



COMFORTABLE SUPPORT AFTER BREAST CANCER SURGERY

JOBST®

DEFY GRAVITY

Bellisse® - exclusively by JOBST®

MEDICAL COMPRESSION GARMENT



Bellisse®

- exclusively by JOBST®

Padded straps are adjustable.

- Comfortable wearing without shoulder pressure

Special Compressure Comfort® inner support system provides circumferential compression.

- Comfort, day and night

The natural stretch cups have an inner pocket to accommodate the lateral pad or the chest-wall pad.

- Compression promotes lymph drainage

Secure front zipper with hooks and eyes to hold garment in place during zipping.

- For easy donning and doffing

Soft, non-rolling lower band keeps garment from riding or rolling up.

- Soft on the skin
- Smooth while sleeping
- Unobtrusive under clothing


COMFORT THAT MAKES A DIFFERENCE FOLLOWING BREAST CANCER SURGERY

An innovative approach to breast and chest wall edema following breast cancer surgery

Bellisse® with its special **Compressure Comfort®** inner support system is the innovative solution for the management of breast and chest wall edema. **Bellisse®** is designed to meet the specific needs of women suffering from lymphedema, post-surgical discomfort, post-radiation edema and /or fibrosis. This medical compression garment can be worn day and night, is fully adjustable, and provides gentle compression with and without the addition of breast pads. It helps to reduce discomfort and provides gentle, even compression around the torso, facilitating lymph drainage from the breast and chest wall.

Bellisse® was developed by Lesli R. Bell, DPT, CLT-LANA, a specialized lymphedema therapist. Her interest in women's health and oncology led her to pursue an idea for a compression garment for the management of lymphedema of the chest wall. **Bellisse®** is the first compression product made especially for the needs of the post breast cancer surgery patient.

Highly adjustable with three position back closure and with extender providing additional 3 rows of eye loops.

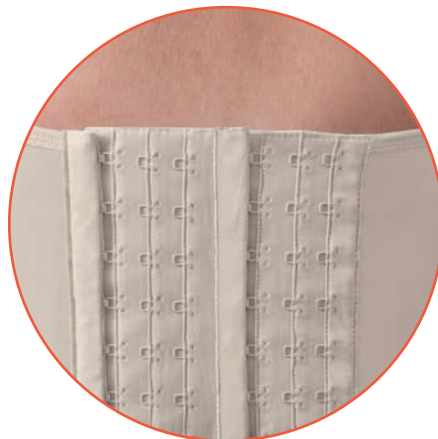

Colours



Buff



Black


MEASURING GUIDELINES

Step 1: Determine your client's Band Size:

- Measure circumferentially below the bust where a regular bra rides.
- Drop the tape 5 cm - 7.5 cm and measure around the bottom of the rib cage.
- Use the larger of the two rib cage measurements for your client's band size. Round odd numbers up.

Step 2: Determine your client's Cup Size:

- Measure circumferentially around the fullest part of the bust.
- Subtract largest ribcage measurement from the bust measurement to determine cup size.

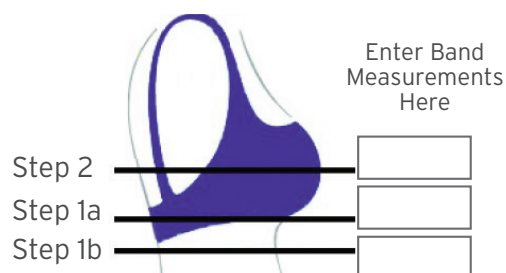

Difference Cup Size

5 cm or less A / B

5 cm - 10 cm C / D

10 cm - 15 cm DD / E

If more than 15 cm go up one band size.

Bellisse Size:

*Bellisse® and Compressure Comfort® are registered trademarks of Lightning2, LLC.

JOBST® JOVIPADS

Available in different shapes, the **JOBST® JoViPads** address particular areas of the breast and chest wall affected by pitting edema and/or fibrosis. The pads provide gentle and prolonged pressure to soften congested tissue and encourage evacuation of lymph fluid to functional pathways. Designed for use under the **Bellisse®** Medical Compression Garment.



Lateral Pad



Mini Axilla Pad



Chest-wall Pocket Pad



Post-Lumpectomy Pad



Serratus Anterior Pad



Unilateral Post-Mastectomy Pad



Double Mastectomy Pad



Axilla Pad



Inframammary Pad



Cleavage Pad



Padded Insert

Bellisse® exclusively distributed by JOBST®

Please refer to the product label and / or package insert for full instructions on the safe use of these products.



JOBST®,
an Essity brand



jobstcanada.com
essity.com

Essity
1275 North Service Road, Suite 800
Oakville, ON Canada L6M 3G4
Tel 1-877-978-5526 Fax 1-877-978-9703

62755E R2 136598E ©Essity Hygiene and Health AB 2023 123