EN TENA Identifi™ – Logger Kit

Supports convenient and effective continence assessment. TENA Identifi™ Logger is a reusable device used to collect and transmit data related to voiding from TENA Identifi™ Sensor Wear products. TENA Identifi™ Sensor Wear products are not included. Use only with charger unit 61408. Latest instruction manual available at <https://tenaidentifi.essity.com>.

1. CHARGING

- First clean and disinfect the Logger with non-corrosive disinfectant.
- Place the Logger in the charger and wait for the status light to turn on.
- Flashing orange = charging.
- Steady green = fully charged.

2. INITIATE DATA COLLECTION

- Go to tenaidentifi.essity.com and initiate new assessment.
- Apply Sensor Wear on the user.
- Insert Logger.
- Green light flashes 15 sec.
- If the status light is orange, the Logger needs to be charged. Follow steps in section 1. Charging above.

3. CHANGING SENSOR WEAR

- Remove the Logger from Sensor Wear.
- Dispose Sensor Wear according to protocol.
- Follow local hygiene routines and then apply fresh Sensor Wear on the user.
- Reinsert the same Logger.

4. REPORT

- After 72 hours, assessment is done.
- Go to <https://tenaidentifi.essity.com> and stop the Logger. Add manual events and then complete the assessment.
- Generate report.
- Analyse and plan continence care.

5. LOGGER SYMBOLS

	The serial number of the Logger.
	The article number of the Logger.
	Read the manual before use.
	The Logger has the CE mark and is therefore safe to use.
	The Logger contains a battery, and must not be disposed in a garbage bin.
	The Logger contains a radio transmitter.
	The Logger is a type BF applied part complying with IEC 60601-1.
	The ingress protection is IP65, meaning the Logger is dust proof and splash proof.
	The Logger identity used to identify the Logger in the web portal https://tenaidentifi.essity.com .
	The legal manufacturer of the Logger.
	Medical device.

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