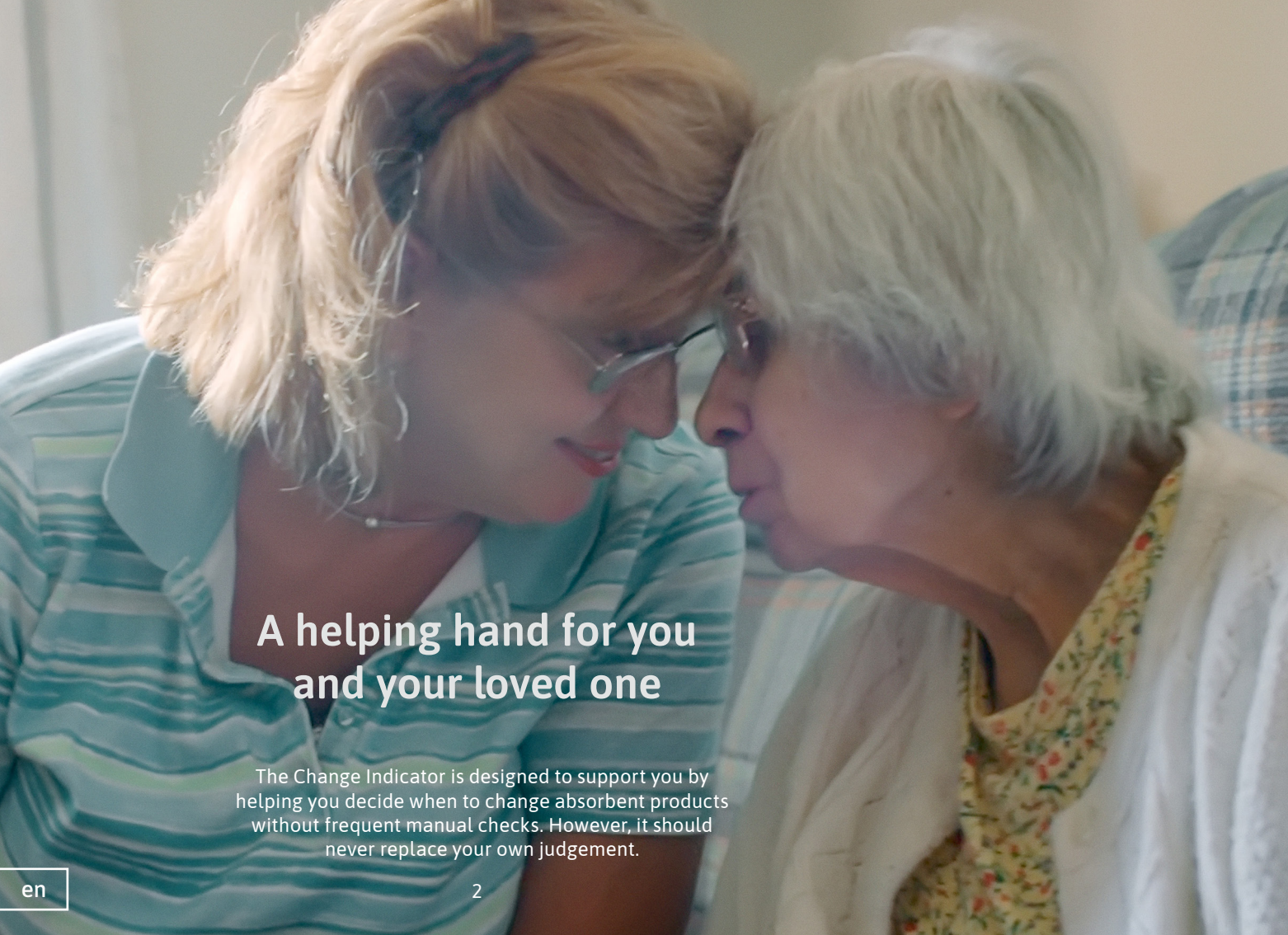
QUICK GUIDE

TENA SmartCare Change Indicator™

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A helping hand for you and your loved one

The Change Indicator is designed to support you by helping you decide when to change absorbent products without frequent manual checks. However, it should never replace your own judgement.

Important information

The Change Indicator is intended for use with TENA incontinence products.

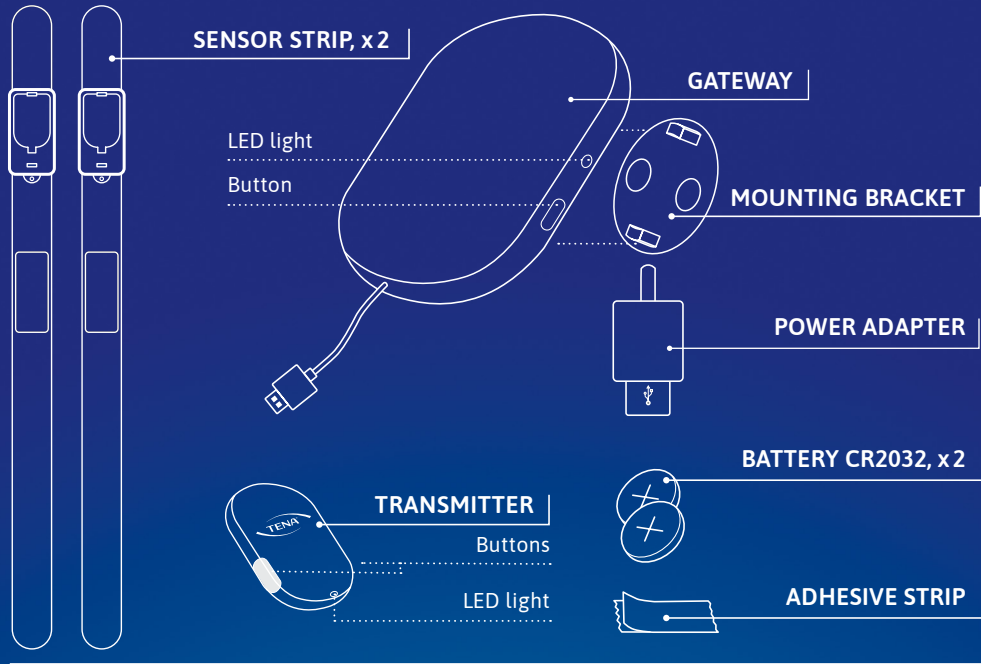
Clean the Change Indicator when fully assembled to prevent water from entering the device. Do not submerge in water or clean in a washing machine.

Do not attempt to repair or modify the Change Indicator or a gateway by yourself.

Seek immediate medical care if the battery is swallowed.

Keep the Change Indicator and batteries away from children and pets.

What's in the box

You also need (not included):

SMARTPHONE
iOS 11/Android 7
or later

1 Start with the app

1.1 DOWNLOAD APP AND REGISTER ACCOUNT

Scanning the relevant QR code will take you directly to the App Store or Google Play, where you can download the TENA SmartCare Family Care app.

To App Store (iOS)

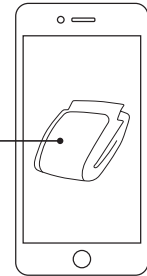
To Google Play (Android)

Once you have downloaded the app, start setting up the Change Indicator using the instruction films in the app or this quick guide.

1.2 REGISTER YOUR LOVED ONE'S PRODUCT IN THE APP

For the most accurate results, register the absorbent product your loved one is currently using in the app.

2 Set up the gateway

CONNECT

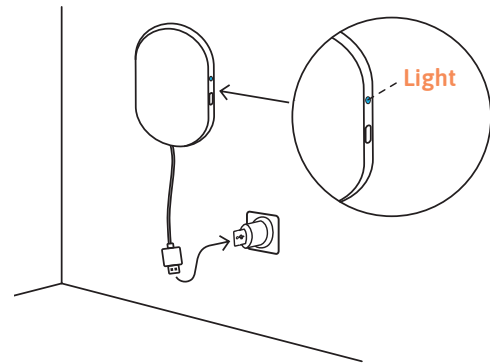
CONSIDER RANGE (33 FT/ 10 M)

The gateway can be mounted on a wall or placed on a flat surface.

2.1 Connect to power outlet


Blinking light
= connecting

Steady light
= connected

No light
= OK

Allow up to 60 seconds for the light to start blinking and another 60 seconds for it to turn green.

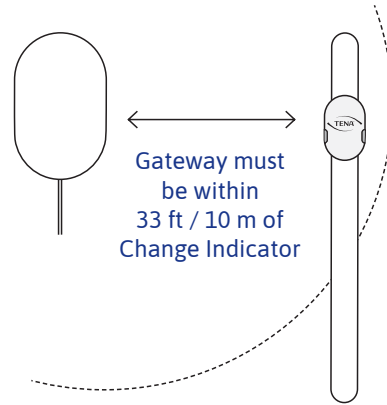
2.2 Consider range

3 Register the transmitter

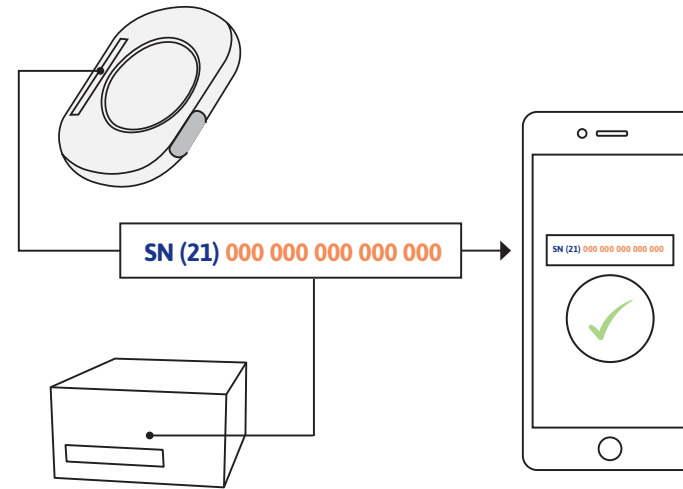On the back of the transmitter, and on the transmitter packaging, there is a 15-digit code. To register the transmitter, enter the 15-digit code in the app. The app will confirm when the code has been approved.

4 Assemble the Change Indicator

INSERT BATTERY IN THE TRANSMITTER

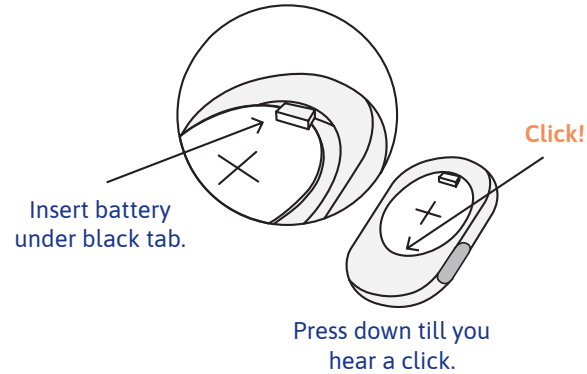
ATTACH TRANSMITTER TO SENSOR STRIP


4.1

4.2

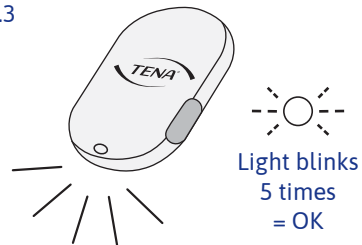
4.3


8

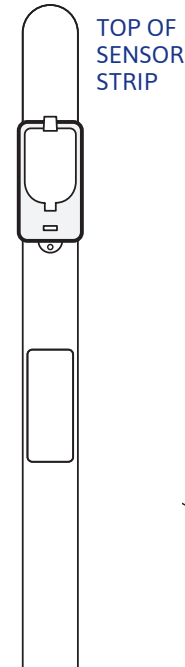
4.4

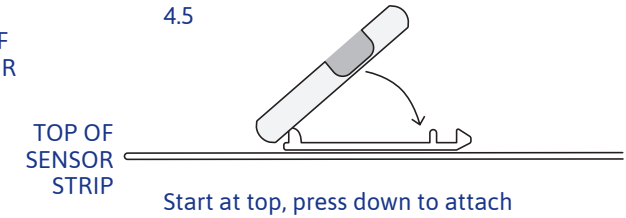
4.5

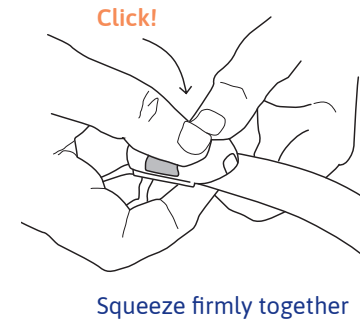4.6

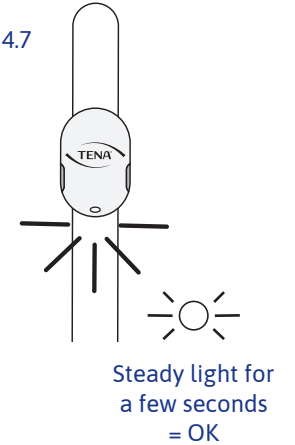4.7

9

5 Attach Change Indicator to product

FIND THE FRONT

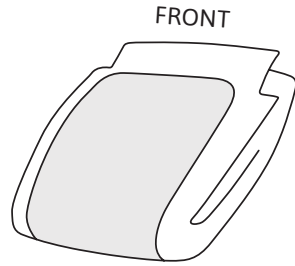
KEEP FOLDED

ALIGN WITH THE TOP OF THE THICKER ABSORBENT AREA

PLACE TIGHTLY

TENA Slip/TENA Flex


5.1

5.2

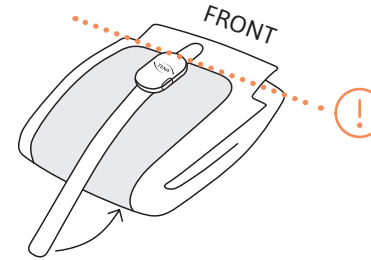
5.3


Press firmly! Tight fit all the way

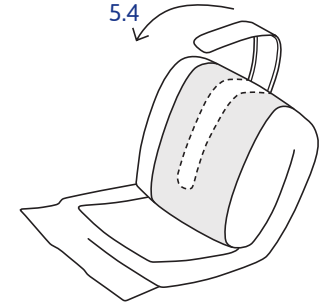5.4


TENA Pants

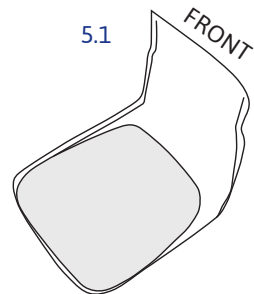
5.1

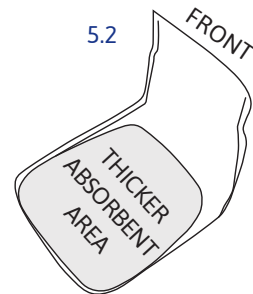5.2

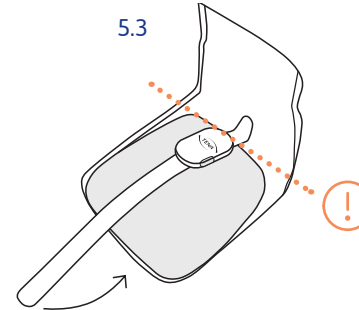
5.3

Press firmly! Tight fit all the way

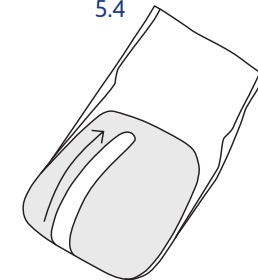5.4

6 Use the Change Indicator

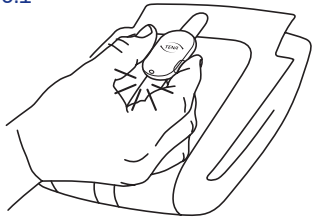
PRESS BUTTONS

APPLY THE TENA PRODUCT

FOLLOW NOTIFICATIONS

CONSIDER CHANGE

6.1


Press both buttons
for 2 seconds.

Steady light for a
few seconds = OK

Press both buttons simultaneously
for two seconds after every change.
This registers the change and
resets the system.

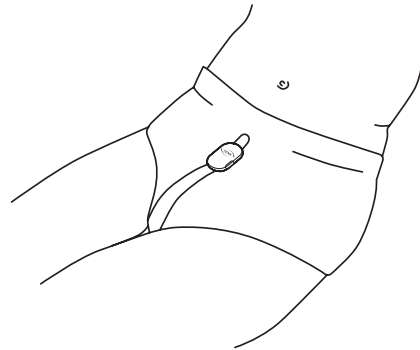6.2

Apply on your loved one.

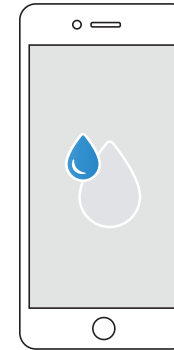
If the user is very mobile,
consider applying a fixation pant
or ordinary underwear on top of
the Change Indicator. This will
keep it comfortably in place.

6.3

DRY
– no action needed
The absorbent part of
the TENA product is dry.

6.4

SOME URINE
– no action needed
Some urine in the absorbent
part of the TENA product.
The surface still feels dry
to the skin.

6.5

WET
– consider change
More urine in the
absorbent part of the TENA
product. It probably still
feels dry to the skin, but you
should consider change.

In case of error notifications, consult the instructions for use for more information.

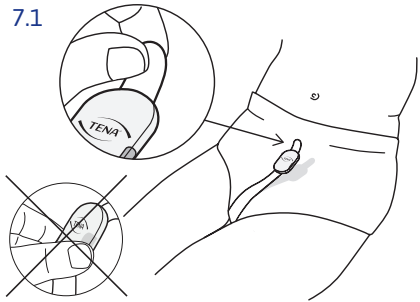
7 Remove and clean

REMOVE THE CHANGE INDICATOR

CLEAN WHEN ASSEMBLED

7.1

Remove Change Indicator before removing the TENA product.

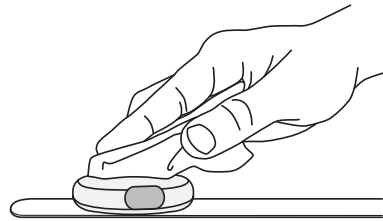
Start at the top. Grip the tab, then gently peel straight downwards.

Place Change Indicator on a flat surface when not in use.

Remember to press both buttons simultaneously for two seconds before or after every change. This registers the change and resets the system.

7.2

Only clean when assembled.

Clean once a week or as needed.

Use an alcohol-based disinfectant when needed.

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en

Ensure that you have read and understood the instructions for use before using the TENA SmartCare Change Indicator™ system. **The instructions for use** are available through the TENA SmartCare Family Care app (Menu > Help > Instruction for use) or www.tena.com/SmartCare/eIFU

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www.tena.com/SmartCare/eIFU

Contact us for support or information
www.tena.com

www.tena.com/SmartCare/eIFU

Essity Hygiene and Health AB
SE-405 03 Göteborg, Sweden
Visiting address: Mölndals Bro 2, Mölndal
www.essity.com

essity

TENA
an Essity brand