

COMPRESSION FOR EMPOWERING SELF-CARE

STRONG & LITE

Easy to use
compression
solutions

JOBST® FarrowWrap®
Adjustable Compression Device



DEFY GRAVITY

EASY-TO-USE COMPRESSION SOLUTIONS HELP TO PROMOTE INDEPENDENT SELF-CARE



CHOOSING THE RIGHT COMPRESSION¹:

- ✓ Easy to apply and remove
- ✓ Is comfortable for the patient and stays in place
- ✓ Offers variety in choice and is aesthetically acceptable

Compression therapy is a vital component in the management of chronic swelling from venous and lymphatic conditions. Conventional treatment focuses on reducing symptoms with multi-layer bandaging.

Give your patients the choice and variety needed to take control of their own care and support them to achieve better therapy outcomes with adjustable compression wraps.

In edema management, adjustable compression wraps can be used as a first-choice treatment option and offer an alternative to multi-layer bandaging or hosiery. They have similar short-stretch properties to multi-layer bandaging in that they have low resting pressure and a high working pressure. They are designed for easy use which enables patients to independently self-apply and adjust the compression according to their needs and tolerance.

DECONGESTION

MAINTENANCE



► **REDUCING THE EDEMA TO A MANAGEABLE SIZE AND IMPROVING SKIN HEALTH**



► **KEEPING EDEMA STABLE OVER TIME BY ADAPTING THERAPY TO NEW NEEDS**



JOBST® FARROWWRAP®

Short stretch compression wraps for day and night

- Easy-to-use adjustable bands
- Easy to don
- For patients requiring independence
- FarrowWrap® is perfect for patients with fluctuating edema who have difficulty donning traditional compression stockings
- Also used for vascular disorders in conjunction with wound care



JOBST® COMPRI2

2-layer short stretch compression systems

- Sustained compression for up to 7 days
- *Targeted nominal compression for an average ankle size.



COMPRILAN®

Short stretch compression bandages

- Ideal when frequent applications are required

DAYTIME



JOBST® ELVAREX®* AND CONFIDENCE

- Custom-made compression garments for upper and lower extremities.
- *Caution: These products are made with natural rubber latex.

NIGHTTIME



JOBST RELAX

- Customized, flat-knitting technology provides consistent gradient compression and stays in place.
- Monofilament spacer layer provides cushioning and promotes airflow for comfort during the night



JOBST® JOVIPAK®

- Custom-made compression garments
- Promotion of lymph drainage
- Softening of fibrosis

THE EASY CHOICE TO SUPPORT PATIENT SELF-CARE AND EMPOWERMENT

Give your patients the tools they need to better manage their swelling and respond confidently to fluctuating edema levels, with a solution that takes the guesswork out of applying compression safely.

JOBST FarrowWrap garments are a range of adjustable compression wraps that are available in a variety of styles, fabrics, sizes and compression classes. Made with durable materials, they are designed to help reduce limb swelling and maintain volume reductions, as well as promote healing of venous leg ulcers. They are designed to provide the same strong support as multi-layer bandaging but in a simplified, intuitive design.

WHICH OF YOUR PATIENTS WOULD BENEFIT FROM A COMPRESSION WRAP?



EDEMA

Patients with swelling resulting from lymphedema and venous edema:


- With fluctuations in limb swelling
- In decongestive and/or maintenance phases



VENOUS LEG ULCER

Patients with open lesions between the knee and the ankle:

- Most common type of leg ulcer, accounting for up to 80% of all leg ulcer cases
- Can develop after a minor injury, where persistently high pressure in the veins of the legs has damaged the skin

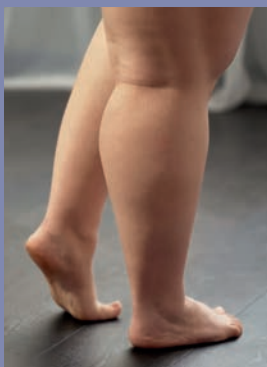

LARGER BODY SIZES

Patients unable to don or unsuitable for compression garments/bandages through:

- Severe obesity
- Overweight or larger body sizes
- Limb distortion due to edema



PHYSICAL LIMITATIONS

Patients unable to apply or remove compression hosiery/bandaging due to:

- Arthritis or weak hand strength
- Back problems
- Sensitive or fragile skin at risk of breakdown



APPLY AND ADJUST CONFIDENTLY

JOBST FarrowWrap was engineered for easy application and can offer all the benefits of multilayer bandaging without the complexity of application. The overlapping material* supports consistent self-application without the need for specialized intervention by healthcare professionals. The overlap provides the support needed to efficiently control edema and help reduce the risk of swelling forming between the bands.

* Overlapping bands not on JOBST FarrowWrap 4000

Overlapping bands* replicate the principles of multi-layer bandaging.

PROVEN PATIENT SATISFACTION*

85%

OF PATIENTS
EASILY ADAPTED
TO INDIVIDUAL
NEEDS

73%

OF PATIENTS
ASSESSED
MANAGEABILITY
POSITIVELY

A clinical study on the performance and safety of JOBST FarrowWrap found that 85% of patients were able to easily adapt the wrap to suit their individual needs. 73% of patients gave a positive assessment of JOBST FarrowWrap as being good or very good in terms of manageability, making them a suitable choice for managing the challenging issues associated with venous and lymphatic conditions.

* Observational Study JOBST® FarrowWrap® 2019, data on file

HELP YOUR PATIENT TO THE RIGHT PRODUCT

JOBST FarrowWrap LITE

For patients with mild to moderate edema.

JOBST FarrowWrap STRONG /
JOBST FarrowWrap CLASSIC

For patients with moderate to severe edema.

JOBST FarrowWrap 4000

For patients with moderate to severe venous and lymphatic conditions with or without a venous leg ulcer present.

JOBST FarrowWrap BASIC

The economically affordable solution for patients with moderate to severe chronic edema.

JOBST® FARROWWRAP®




Easy Donning

Easy-to-attach VELCRO® brand tabs support easy donning and doffing.



Short Stretch
Technology

Short-stretch technology provides low resting and high working pressures to enhance lymphatic and venous return.

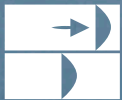

Adjustable

VELCRO® brand hook & loop system allows patients to respond better to fluctuating edema by simply tightening or loosening the wrap.



Low Profile
Design

Discreet design that's less bulky than multi-layer bandaging and can be worn under normal clothing.






JOBST® FARROWWRAP® LITE

Contains a lower level of compression than other FarrowWrap products and is suitable for patients with mild to moderate edema.

- Support patient when reduced compression is required
- Adapts to limb shape
- Liner included

COLOUR

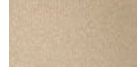

Beige

COMPRESSION CLASS

20 - 30 mmHg

STYLES



Footpiece^{1,2,3}



Legpiece^{1,2,3}



Kneepiece^{1,***}
Thighpiece^{1,2,**}



Armpiece^{1,2,3}



Ambidextrous
Hand Gauntlet¹



JOBST® FARROWWRAP® STRONG

Made from durable fabric with a soft inner layer.
Recommended for patients with moderate to severe edema.

- Reliable hold for fluctuating edema
- Liner included
- More rigid than JOBST FarrowWrap LITE, less stiff than JOBST FarrowWrap CLASSIC

COLOUR



Beige

COMPRESSION CLASS

30 - 40 mmHg


STYLES



Footpiece^{1,2,3}



Legpiece^{1,2,3}

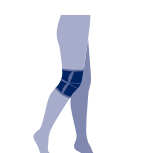

Kneepiece¹



Thighpiece^{1,2,*}



Armpiece²

1: RTW (Ready-To-Wear) | 2: Made to Measure (MTM) | 3: TTF (Trim-To-Fit)

* Available for RTW (Ready-To-Wear) and Made to Measure (MTM) ** Thighpiece must be worn with Kneepiece *** STRONG kneepiece can be combined with CLASSIC garment

JOBST® FARROWWRAP® CLASSIC

Made from more rigid material than JOBST FarrowWrap STRONG and recommended for patients with moderate to severe edema.

- Designed to sit flat against skin folds and not dig into the skin
- Liner included
- Stiffer than JOBST FarrowWrap STRONG

COLOUR



Beige

COMPRESSION CLASS

30 - 40 mmHg

STYLES



Footpiece^{1,2,3}



Legpiece^{1,2}



Kneepiece^{1,***}



Thighpiece^{1,2,**}



Armpiece²

JOBST® FARROWWRAP® 4000

Made with an inner sleeve to aid donning. Ideal for patients with moderate to severe venous and lymphatic conditions with or without a venous leg ulcer present.

- Designed for easy application with just four bands
- Helps promote healing of venous ulceration
- Comes with a JOBST FarrowHybrid compression sock

Note: Open wounds must be covered with an appropriate dressing before the compression garment is put on.

COLOUR



Beige

COMPRESSION CLASS

30 - 40 mmHg

STYLE



Legpiece¹



JOBST® FARROWWRAP® BASIC

The economically affordable solution for patients with moderate to severe chronic edema and regular limb shapes.

- Removable, double-sided VELCRO® brand tabs
- Band length can be changed and trimmed as limb size reduces
- Comes with JOBST FarrowHybrid compression sock

COLOUR



Beige

COMPRESSION CLASS

30 - 40 mmHg

STYLES



Legpiece¹

1: RTW (Ready-To-Wear)

JOBST® FARROWWRAP® ACCESSORIES



JOBST® FARROWWRAP® LINER

Above the knee liner for FarrowWrap legpieces and foot pieces.



ARM LINER

Liner for Trim-To-Fit and Custom arm pieces.



FARROWHYBRID ADI FOOT COMPRESSION SOCK


Below knee high liner with 20-30 mmHg of compression in the foot area.

ARMPIECE

JOBST® FarrowWrap® LITE

Description	Colour	Length	XS	S	M	L	XL
RTW Armpiece Right	Tan	Short		7665012	7665014	7665016	
RTW Armpiece Left	Tan	Short		7665011	7665013	7665015	
RTW Armpiece Right	Tan	Regular		7665006	7665008	7665010	
RTW Armpiece Left	Tan	Regular		7665005	7665007	7665009	
RTW Armpiece Right	Tan	Long		7665000	7665002	7665004	
RTW Armpiece Left	Tan	Long		7665035	7665001	7665003	
Hand Gauntlet with Foam Squares	Tan	Regular	7665300	7665301	7665302	7665303	
TTF Armpiece Right (includes Velcro)	Tan	Regular			7612002		7612003
TTF Armpiece Left (includes Velcro)	Tan	Regular			7612000		7612001
TTF Armpiece Extra Velcro	7612004						

SIZING CHART - JOBST® FARROWWRAP® ARMPIECE

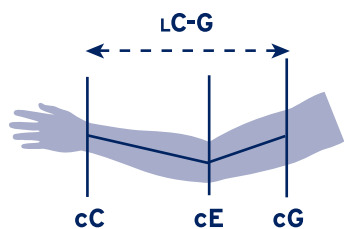

Size	S	M	L
cG	22-31 cm	29-39 cm	32-45 cm
cE	20-27 cm	25-34 cm	30-40 cm
cC	14-18 cm	16-21 cm	19-25 cm
Length (cm)	Short	Regular	Tall
LC-G	40-43 cm	44-47 cm	48-51 cm



LITE RTW



CLASSIC CUSTOM



**LITE CUSTOM STRONG
CUSTOM LITE TTF
(ADJUSTABLE)**

THIGHPIECE

JOBST® FarrowWrap® LITE

Description	Colour	Length	XS	S	M	L	XL
RTW Thighpiece	Tan	Short	7665505	7665506	7665507	7665508	7665509
RTW Thighpiece	Tan	Regular	7665500	7665501	7665502	7665503	7665504
RTW Thighpiece	Tan	Long	7665510	7665511	7665512	7665513	7665514


JOBST® FarrowWrap® CLASSIC

Description	Colour	Length	XS	S	M	L	XL
RTW Thighpiece	Tan	Short	7665806	7665807	7665808	7665809	7665810
RTW Thighpiece	Tan	Regular	7665800	7665801	7665802	7665803	7665804
RTW Thighpiece	Tan	Long	7665812	7665813	7665814	7665815	7665816

JOBST® FarrowWrap® STRONG

Description	Colour	Length	XS	S	M	L	XL
RTW Thighpiece	Tan	Short	7664905	7664906	7664907	7664908	7664909
RTW Thighpiece	Tan	Regular	7664900	7664901	7664902	7664903	7664904
RTW Thighpiece	Tan	Long	7664910	7664911	7664912	7664913	7664914

SIZING CHART - JOBST® FARROWWRAP® THIGHPIECE

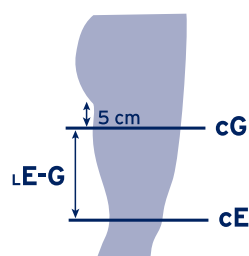

Size	XS	S	M	L	XL
cG	60-70 cm	65-75 cm	70-80 cm	75-90 cm	85-100 cm
cE	47-52 cm	53-59 cm	60-66 cm	67-73 cm	74-80 cm
Length (cm)	Short		Regular		Tall
L E-G*	16-20 cm		21-26 cm		27-31 cm

Thighpiece comes with FarrowWrap® Kneepiece



LITE, STRONG



CLASSIC

*E-G: Medial length from just above patella to the groin. Follow contour of any skin folds/lobules.
Most thighpieces can accommodate a 20% reduction of the circumference.

FOOTPIECE

JOBST® FarrowWrap® LITE

Description	Colour	Length	XS	S	M	L	XL
RTW Footpiece	Tan	Regular	7665205	7665206	7665207	7665208	7665209
RTW Footpiece	Tan	Long	7665200	7665201	7665202	7665203	7665204
TTF Footpiece (includes Velcro)	Tan	Regular			7612100		
TTF Footpiece Extra Velcro	7667500						


JOBST® FarrowWrap® CLASSIC

Description	Colour	Length	XS	S	M	L	XL
RTW Footpiece	Tan	Regular	7665605	7665606	7665607	7665608	7665609
RTW Footpiece	Tan	Long	7665600	7665601	7665602	7665603	7665604

JOBST® FarrowWrap® STRONG

Description	Colour	Length	XS	S	M	L	XL
RTW Footpiece	Tan	Regular	7664604	7664605	7664606	7664607	7664608
RTW Footpiece	Tan	Long	7664609	7664600	7664601	7664602	7664603
TTF Footpiece (includes Velcro)	Tan	Regular			7611600		
TTF Footpiece Extra Velcro	7611601						

SIZING CHART - JOBST® FARROWWRAP® FOOTPIECE

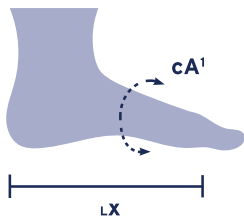

Size	XS	S	M	L	XL
cA¹	22-24 cm	25-27 cm	28-30 cm	31-34 cm	35-40 cm
LX Regular*	16-17 cm	17-18 cm	19-20 cm	20-21 cm	22-23 cm
LX Long**	18-19 cm	19-20 cm	21-22 cm	22-23 cm	24-25 cm



CLASSIC



LITE, STRONG

*Circumferences measurements must be within ± 0.5 cm

**Length from back of the heel to base of pinky toe

LEGPIECE

JOBST® FarrowWrap® LITE

Description	Colour	Length	XS	S	M	L	XL
RTW Legpiece	Tan	Regular	7665400	7665401	7665402	7665403	7665404
RTW Legpiece	Tan	Long	7665405	7665406	7665407	7665408	7665409
TTF Legpiece (includes Velcro)					7612300		
TTF Legpiece Extra Velcro	7612301						
TTF Legpiece Extra Band	Tan	Regular			7612302		

JOBST® FarrowWrap® CLASSIC

Description	Colour	Length	XS	S	M	L	XL
RTW Legpiece	Tan	Regular	7665700	7665701	7665702	7665703	7665704
RTW Legpiece	Tan	Long	7665705	7665706	7665707	7665708	7665709

JOBST® FarrowWrap® STRONG


Description	Colour	Length	XS	S	M	L	XL
RTW Legpiece	Tan	Regular	7664800	7664801	7664802	7664803	7664804
RTW Legpiece	Tan	Long	7664805	7664806	7664807	7664808	7664809
TTF Legpiece (includes Velcro)	Tan	Regular			7611800		7611801
TTF Legpiece Extra Velcro	7612301						
TTF Legpiece Extra Band	Tan	Regular			7611803		7611804

JOBST® FarrowWrap® BASIC*

Description	Colour	Length	XS	S	M	L	XL
RTW Legpiece (includes Velcro)	Tan	Regular	7666000	7666001	7666002	7666003	7666004
RTW Legpiece (includes Velcro)	Tan	Long	7666005	7666006	7666007	7666008	7666009

*Comes with a JOBST® FarrowHybrid® ADI Wide Foot Compression Liner

SIZING CHART - JOBST® FARROWWRAP® LEGPIECE

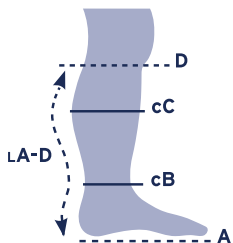

Size	XS	S	M	L	XL
cC	36-43 cm	42-50 cm	48-58 cm	53-63 cm	58-68 cm
cB	21-25 cm	25-30 cm	30-36 cm	36-42 cm	42-50 cm
LA-D Regular*	33-37 cm	35-39 cm	37-41 cm	39-43 cm	39-43 cm
LA-D Tall*	38-41 cm	40-43 cm	42-45 cm	44-47 cm	44-47 cm

Circumferences measurements must be within ± 1.0 to 1.5 cm

*A-D: Floor length up to 2 fingers below knee fold


LEGPiece

JOBST® FarrowWrap® 4000*

Description	Colour	Length	XS	S	M	L
Legpiece	Tan	Regular	7666100	7666101	7666102	7666103
Legpiece	Tan	Long	7666104	7666105	7666106	7666107
Legpiece	Black	Regular	7666108	7666109	7666110	7666111
Legpiece	Black	Long	7666112	7666113	7666114	7666115

*Comes with a JOBST® FarrowHybrid® ADI Wide Foot Compression Liner

SIZING CHART - JOBST® FARROWWRAP® 4000

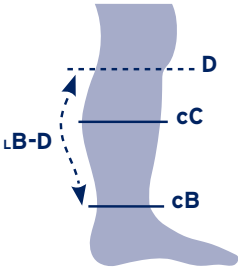

Size	XS	S	M	L
cC	25-38 cm	32-48 cm	37-56 cm	43-60 cm
cB	17-23 cm	19-28 cm	25-37 cm	29-43 cm
L B-D Regular*	30-33 cm	30-33 cm	30-33 cm	30-33 cm
L B-D Tall*	34-37 cm	34-37 cm	34-37 cm	34-37 cm

*B-D: Actual height of the garment



LITE, STRONG



BASIC



CLASSIC



4000

ACCESSORIES

JOBST® FarrowHybrid®

Description	Colour	S	M	L	XL
ADI Regular	Tan	7666400	7666401	7666402	7666403
ADI Large	Tan	7666412	7666413	7666414	7666415
ADI Large	Black	7666416	7666417	7666418	7666419

Calf Circumference must be less than 60 cm

SIZING CHART - JOBST® FARROWHYBRID® ADI



Measurement (cm)	Regular	Large
Mid-foot circumference	20-27	28-35

	S	M	L	XL
Men Shoe Size	3-6	7-9	10-12	13-15
Women Shoe Size	4-7	8-10	11-13	14-16

Do not apply the JOBST® FarrowHybrid® if calf circumference exceeds 60cm.
Smooth out any wrinkles in the garment.

Farrow Standard Liners

All Standard Liners and TG® Soft liners are sold as pairs. Free Knee High liner issued with any legpiece or footpiece & legpiece combination.

Description	Colour	S	M	L
Knee High	Tan	7666621		7666622
Arm	Tan	7666620		

TG® Soft Liners

Description	Colour	S	M	L
Knee High	Tan	7666900	7666902	7666904
Thigh High	Tan	7667000	7667002	7667004
Arm	Tan	7667006	7667008	7667010

SIZING CHART - JOBST® FARROWWRAP® LINER


Measurement (cm)	S	M	L
TG® Soft			
Arm, calf or groin circumference	<40	41-70	71-120
Flesh Tone Liners			
Knee High	19-48 cm		48-70 cm
Arm	19-48 cm		48-70 cm

JOBST® FarrowWrap® LITE Gauntlet

Description	XS	S	M	L
Gauntlet	7665300	7665301	7665302	7665303



LITE GAUNTLET

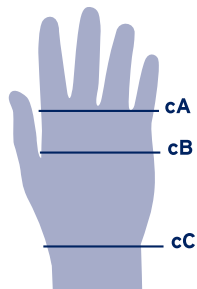

SIZING CHART - JOBST® FARROWWRAP® GAUNTLET

Size	XS	S	M	L
cA	16-18 cm	18-20 cm	20-22 cm	22-24 cm
cB	17-19 cm	19-21 cm	21-23 cm	23-26 cm
cC	15-18 cm	17-20 cm	19-22 cm	22-24 cm

GARMENT CARE

JOBST® FARROWWRAP®



Machine
wash 30°



Hand-
wash



Do not
bleach



Do not
tumble dry



Lay flat
to dry



Do not
iron



Do not
dry clean

DONNING JOBST® FARROWWRAP®




Watch our donning videos to learn more on how to apply JOBST FarrowWrap. Scan the QR code to visit the JOBST website for an overview of all donning videos.




ORDERING JOBST® FARROWWRAP®

JOBST FarrowWrap can easily be ordered online through our JOBST eShop. Scan the QR code to visit the shop and to download the ordering forms.

VISIT JOBST® ESHOP

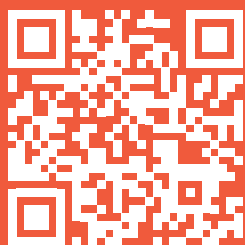

DOWNLOAD ORDER FORMS

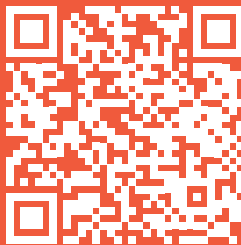

You can also visit jobstcanada.com for all the information you need including Order Forms.



JOBST®,
an Essity brand

 jobstcanada.com essity.com

Essity
1275 North Service Road West, Suite 800
Oakville, ON Canada L6M 3G4
Tel. 1-877-978-5526 Fax 1-877-978-9703
136770E ©Essity Hygiene and Health AB, 2025 125