



COMPRESSION THAT STANDS UP TO LYMPHEDEMA

DEFEY GRAVITY

JOBST

Help your patients stand up to lymphedema with
a **complete range of compression products** designed
with them in mind.

THE COMPLETE SOLUTION FOR LYMPHEDEMA

You are passionate about finding the right products for your patients, as you don't want them to be held down by their condition. JOBST® offers a complete portfolio of products to support you and your patients.

Why JOBST®?	3	JOBST® Custom Seamed.....	13
Compression Bandages		JOBST® SoftFit™ Band Variations.....	14
JOBST® Compri2		Functional Zones for JOBST® Elvarex®,	
Advanced Wound Care.....	4	JOBST® Elvarex® Soft and JOBST® Confidence.....	15
JOBST® FarrowWrap®.....	5	JOBST® Relax	16
JOBST® Daytime Garments Decision Tree.....	6	JOBST® JoViPak®	
JOBST® Elvarex®.....	7	JOBST® JoViPads®	
JOBST® Elvarex® Soft.....	8	JOBST® JoViLiner Arm and Leg	
JOBST® Confidence.....	9	JOBST® PitPaks.....	17
JOBST® Elvarex® Plus		Bellisse® Breast	
JOBST® Elvarex® Soft Seamless.....	10	Chest Wall Pads	18
JOBST® Custom Seamless Soft		JOBST® Donning Aids & Accessories	19
JOBST® Bellavar®.....	11	JOBST® Custom eShop	20
JOBST® Bella™ Lite			
JOBST® Bella™ Strong	12		



WHY JOBST®?



TRUST, CONFIDENCE AND HISTORY

- Conrad Jobst, mechanical engineer and industry pioneer, dedicated his life and expertise to inventing graduated compression garments in 1950.
- JOBST® is a brand of medical compression legwear for the prevention, treatment and management of venous and lymphatic conditions.
- After his passing, Conrad's wife Carolyn, assumed the presidency of the company for the next 27 years. Inspired by her late husband's legacy, Carolyn grew the business into a global organization and extended sales to virtually every continent as the JOBST® reputation for quality grew.

INNOVATION

- JOBST® incorporates customer and healthcare professional involvement in the development of products. From focus groups to wash-and-wear tests, all garments go through a rigorous design process that ensures they provide the highest efficacy, safety, and satisfaction for our customers.
- Innovation is at the heart of our product excellence. Product development efforts are directed towards delivering the highest quality product with the highest assurance of medical efficacy.



EDUCATION

- PATH is a gold standard education platform with a diverse selection of therapy areas and learning styles, to help you provide the best possible patient care.
- Our product experts, called Medical Advisors, and our devoted team of Sales Professionals, provide customized education and solutions for both fitters and therapists.



TRUSTED BRAND

- JOBST® is the #1 physician-recommended brand of medical compression ready-to-wear legwear in the US* for the prevention, treatment and management of venous and lymphatic conditions.

COMPRESSION BANDAGES

MULTI-LAYER BANDAGING



Comprilan®

Short stretch compression bandage

- Provides high working pressure and low resting pressure
- Easy care and durable



Elastomull®

Soft comfortable bandage

- Precision fit in areas with constant movement
- Air-permeable and comfortable to wear



Tricofix®

Lightweight stockinette

- Protects the skin surface under compression, casts, or lymphedema wrapping
- 100% cotton



CompriFoam®

Open cell foam bandage

- 100% polyurethane foam
- Reusable

JOBST® COMPRI2 2-Layer Bandaging

The JOBST® Compri2 is a 2-Layer Bandage System* specifically designed for easy application. It provides graduated and reliable compression for up to seven days to help achieve effective edema reduction and helps promote wound healing.



1st Layer - Non Woven Padding

- **Stretchable** - simplifies application
- **Active Moisture Management** - helps keep skin dry and comfortable
- **Customized Padding** - allows for anatomical adjustments and additional protection to sensitive areas



2nd Layer - Short Stretch Bandage

- **Cohesive** - delivers reliable compression
- **Adjustable** - allows modification during application
- **Visual** - indicator offers simple and effective application support

*As JOBST® Compri2 does not include a wound dressing, any wounds must be treated with a suitable wound dressing before the compression bandage is applied.

ADVANCE WOUND CARE

Cutimed offers a wide range of advanced wound care products for a variety of wound phases and exudate levels that result in effective wound healing.



Cutimed® Siltec®

is a silicone coated, sterile, absorbent polyurethane foam dressing for atraumatic dressing changes that contains superabsorbent stripes which absorb and lock wound exudate.



Cutimed® Sorbion®

is a sterile, hydroactive, gel-forming, superabsorbent wound dressing.



Cutimed® Siltec® Sorbact®

is a bacteria and fungi binding wound dressing, based on Sorbact Technology. It consists of a Sorbact DACCTM coated wound contact layer, combined with an absorbent polyurethane foam that contains superabsorbent stripes and a border with silicone adhesive.



Cutimed® Sorbion® Sorbact®

is a bacteria and fungi binding wound dressing, based on Sorbact Technology. It consists of a Sorbact DACCTM coated wound contact layer combined with a superabsorbent core, which absorbs and retains excessive exudate levels, thereby optimizing the wound environment.

COMPRESSION WRAPS

JOBST® FARROWWRAP®


A FarrowWrap® style to suit every need

JOBST® FarrowWrap® STRONG (30-40 mmHg*)


Premium fabric with strong compression and a soft, inner layer suitable for patients with good limb shape. Recommended for patients with moderate to severe edema. Available in Trim-to-Fit.




Footpiece^{1,2,3}

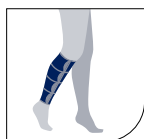


Legpiece^{1,2,3}

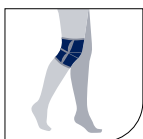

Kneepiece¹

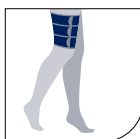

Thighpiece^{1,2,4}

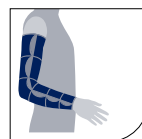

Armpiece (Custom)

Colour


Tan

JOBST® FarrowWrap® CLASSIC (30-40 mmHg*)


Made from more rigid material than JOBST® FarrowWrap® STRONG. Ideal for those with large lobules, skin folds or shape distortions.




Footpiece^{1,2}

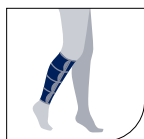

Legpiece^{1,2}

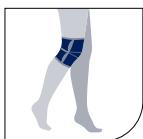

Kneepiece^{1,5}

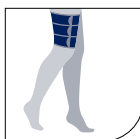

Thighpiece^{1,2,4}



Armpiece² (Custom)


Colour

Tan

Improved VELCRO® with rounded corners. Landing strip at each strap.

JOBST® FarrowWrap® BASIC (30-40 mmHg*)

A value fabric with quality compression. The double-sided VELCRO® enables the band length to be trimmed as the limb reduces.

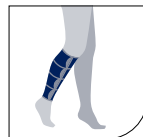


Legpiece¹ (Includes FarrowHybrid Sock)

Colour

Tan

JOBST® FarrowWrap® 4000 (30-40 mmHg*)

JOBST® FarrowWrap® 4000 is ideal for patients with moderate to severe venous and lymphatic conditions with or without a venous leg ulcer. The inner sleeve and 4-band juxtaposing design make for easy application. Anatomically contoured to help prevent gapping.

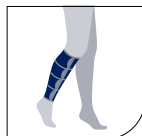

Legpiece¹ (Includes FarrowHybrid Sock)

Colour

Tan

JOBST® FarrowWrap® LITE (20-30 mmHg*)

A double-laminated, durable and breathable fabric especially suitable for home care use/self-management. Available in Trim-to-Fit.



Footpiece^{1,2,3}




Legpiece^{1,2,3}



Kneepiece^{1,5}



Thighpiece^{1,2,4}

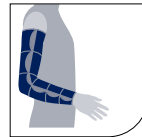

Armpiece^{1,2,3}

Colour

Tan



Hand Gauntlet**

The JOBST® FarrowWrap® Hand Gauntlet manages swelling in the hand and can be used on either the left or right hand. Includes removable 4 mm and 8 mm hook and loop fasteners.



FarrowHybrid ADI Foot Compression Sock

Below knee high liner with 20-30 mmHg of compression in the foot and ankle area. Included with BASIC and 4000 Legpiece.



Easy Donning

Easy-to-attach VELCRO® brand tabs support easy donning and doffing.



Short-Stretch Technology

Short-Stretch Technology provides low-resting and high-working pressures to enhance lymphatic and venous return.



Adjustable

VELCRO® brand hook & loop system allows patients to respond better to fluctuating edema by simply tightening or loosening the wrap.



50% Band Overlap

Optimal width of bands - combined with 50% overlap of bands from bottom to top - provide ideal compression.



Liner Options

- Standard Liner
- TG Soft Liner
- FarrowHybrid Liner

Additional Accessories

- Garmet Grip
- VELCRO® Pack
- Short-Strech Wrap Extra Band

*The average compression for a medium sized ankle.

**Hand Gauntlet for managing mild to moderate edema.

1. RTW (Ready-To-Wear) 2. CM (Custom Made) 3. TTF (Trim-To-Fit) 4. Thighpiece must be worn with Kneepiece 5. STRONG kneepiece can be combined with CLASSIC garment

FLAT-KNIT DECISION TABLE:

JOBST® Confidence vs JOBST® Elvarex® Soft vs JOBST® Elvarex®

Body Shape			
Stage	Mild (I)	Moderate (II)	Severe (III)
Patient Description	First time user, newly diagnosed, preventative treatment, Lipedema/ Lymphedema	First time user, low tolerance to stiff fabric, desire to “feel” the compression, Lipedema/ Lymphedema	Persistent/aggressive edema, Lymphedema (not best for Lipedema)
Skin Folds	None	Minimal	Significant
Tissue	Proportionate, symmetrical, athletic, controlled/maintained edema	Adipose present, beginning development of fibrosis	Significant adipose and/or hard, fibrotic
User Need	Light & Soft material	Soft & Flexible material	Firm & Strong material
JOBST® Product Recommendation	JOBST® Confidence		JOBST® Elvarex® JOBST® Elvarex® Soft JOBST® Elvarex®

DAYTIME GARMENTS

JOBST® ELVAREX®

JOBST's Gold Standard

JOBST® Elvarex® represents JOBST's gold standard in custom daytime garments for the management of lymphedema. These flat-knit garments offer the very best in comfort and containment for even the most severe cases. With a natural rubber core*, Elvarex® provides unsurpassed structure and control for patients of all shapes and sizes.

KEY FEATURES

- Flat-knit structure bridges skin folds instead of working into them
- Natural rubber latex core provides superior containment and allows garment to return to original shape for extended wear
- Excellent breathability for comfort and patient compliance
- Seven compression classes for optimal therapeutic outcomes
- Vast product assortment covers a diverse range of specifications and requirements of patients
- SPF 80 UV protection



EXTREMITIES/COLOURS


Upper Limb

Beige	Black	Navy	Caramel	Cranberry	Cherry	Hazelnut	Bronze
-------	-------	------	---------	-----------	--------	----------	--------

Lower Limb

Beige	Black	Navy	Caramel	Cranberry	Grey	Cocoa	Bronze
Denim**	Henna**	Graphite**					

STYLES

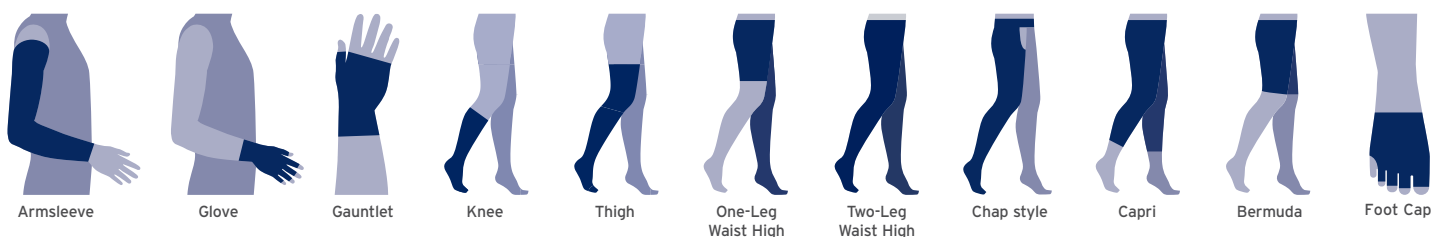

COMPRESSION CLASSES

Upper Limb

CCL 1 ▶ 15-21 mmHg
CCL 2 ▶ 23-32 mmHg
CCL 2 Forte ▶ 23-32 mmHg

Lower Limb

CCL 1 ▶ 18-21 mmHg	CCL 3 Forte ▶ 34-46 mmHg
CCL 2 ▶ 23-32 mmHg	CCL 4 ▶ 49-70 mmHg
CCL 3 ▶ 34-46 mmHg	CCL 4 Super ▶ 60-90 mmHg



*CAUTION: These products contain natural rubber latex which may cause allergic reactions.

**Not available in Foot Cap

DAYTIME GARMENTS

JOBST® ELVAREX® SOFT

Gentle and Not Made with Natural Rubber Latex

JOBST® Elvarex® Soft is the solution for patients wanting a softer, gentler compression garment. The flat-knit fabric allows for easier donning and meets the specific needs of many elderly and pediatric patients, or patients with lipedema. While soft and forgiving, Elvarex® Soft still provides the containment that mild to moderate lymphedema patients need.



EXTREMITIES/COLOURS


Upper Limb

Beige	Black	Navy	Cocoa	Grey	Cranberry	Cherry	Honey
-------	-------	------	-------	------	-----------	--------	-------

Lower Limb

Beige	Black	Navy	Cocoa	Grey	Cranberry	Cherry
-------	-------	------	-------	------	-----------	--------

STYLES

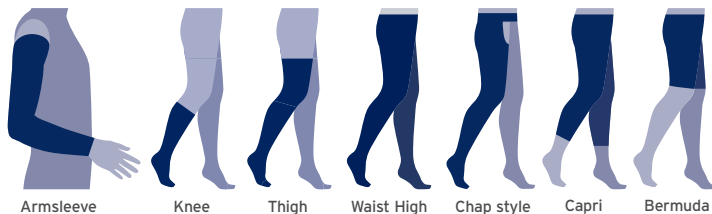

COMPRESSION CLASSES

Upper Limb

CCL 1 ▶ 15-21 mmHg
CCL 2 ▶ 23-32 mmHg

Lower Limb

CCL 1 ▶ 18-21 mmHg
CCL 2 ▶ 23-32 mmHg
CCL 3 ▶ 34-46 mmHg



DAYTIME GARMENTS

JOBST® CONFIDENCE

Next Generation of Made-to-measure Flat-knit Compression Garments


JOBST® Confidence helps your patients feel unrestricted, with a one-of-a-kind fit garment that's as unique as they are. The new generation of flat-knit technology used in JOBST® Confidence allows for a precise fit to the real anatomical body shapes. This personally refined fit not only provides comfort, but also supports your patients with a greater degree of movement.

KEY FEATURES


- 3D innovative knitting method allows the garment to conform to individual body shapes
- Flexible, conforming fabric makes it easy to put on and take off, while providing firm comfort
- Moisture wicking with double-layer construction for optimal comfort
- Not made with natural rubber latex



COLOURS

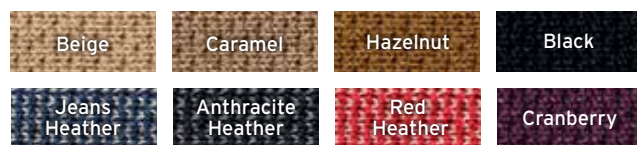


MOTIVATIONAL WAISTBAND

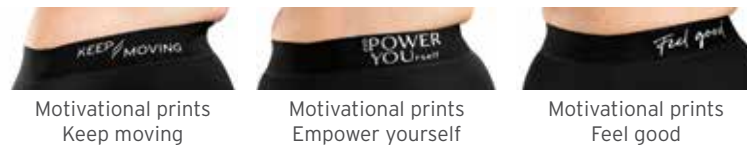


SEAMS



DESIGN OPTIONS

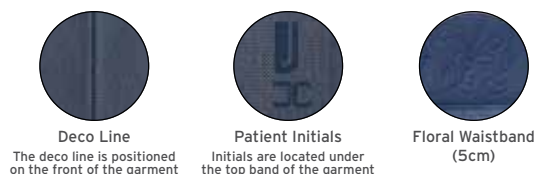

COMFORT OPTIONS

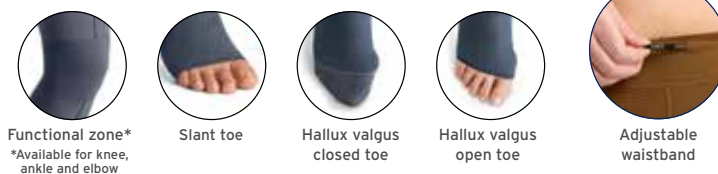

STYLES



COMPRESSION CLASSES

Upper Limb

CCL 1 ▶ 15-21 mmHg

CCL 2 ▶ 23-32 mmHg

Lower Limb

CCL 1 ▶ 18-21 mmHg

CCL 2 ▶ 23-32 mmHg

CCL3 ▶ 34-46 mmHg

PATIENTS PREFER THE COMFORT OF JOBST® CONFIDENCE

RANGE OF MOTION:

85% of patients said they can move freely with JOBST® Confidence.

MOISTURE MANAGEMENT:

92% of patients agreed JOBST® Confidence keeps the skin dry.

GENERAL WEARING COMFORT:

84% of patients evaluated the comfort of JOBST® Confidence as good/very good.

PATIENT SATISFACTION:

7/10 patients said they would recommend JOBST® Confidence.

DAYTIME GARMENTS

JOBST® ELVAREX® PLUS*

The Cutting Edge of Flat-Knit Technology

JOBST® Elvarex® Plus uses advanced flat-knitting technology to construct garments in three dimensions to more closely conform to the shape of patients' hands and feet. These seamless garments help minimize potential irritation between the fingers and toes, and at the base of the thumb, help patients with the comfort they want and the performance they need.

*Caution: This product contains natural rubber latex which may cause allergic reactions.



EXTREMITIES/COLOURS


Upper Limb

Beige	Black	Navy	Caramel	Cranberry	Red	Hazelnut	Cherry	Bronze
-------	-------	------	---------	-----------	-----	----------	--------	--------

Lower Limb

Beige	Black	Navy	Caramel	Cranberry	Red	Grey	Cocoa	Bronze
-------	-------	------	---------	-----------	-----	------	-------	--------

STYLES

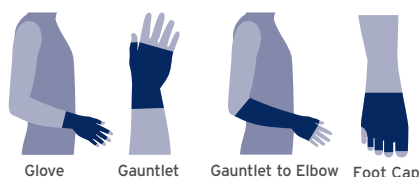

COMPRESSION CLASSES

CCL 1 ▶ 15-21 mmHg (Glove/Gauntlet)
CCL 1 ▶ 18-21 mmHg (Foot Cap)
CCL 2 ▶ 23-32 mmHg



JOBST® ELVAREX® SOFT SEAMLESS

Soft and Seamless, Where it Matters Most


JOBST® Elvarex® Soft Seamless combines all of the benefits of Elvarex® Soft with the seamless construction that patients want in a glove or toe cap. Not only are these flat-knit garments soft and not made with natural rubber latex, they are seamless, improving comfort by helping to eliminate potential irritation between the fingers and toes.



KEY FEATURES

- Soft, durable, and breathable flat-knitted fabric
- Flat edge finish at fingers and toes decreases bulk and prevents curling
- Easy to don, comfortable to wear

STYLES

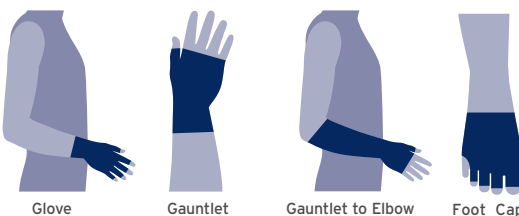

COMPRESSION CLASSES

CCL 1 ▶ 15-21 mmHg (Glove/Gauntlet)
CCL 1 ▶ 18-21 mmHg (Foot Cap)
CCL 2 ▶ 23-32 mmHg

EXTREMITIES/COLOURS

Upper Limb

Beige	Black	Navy	Cocoa	Grey	Cranberry	Cherry	Honey
-------	-------	------	-------	------	-----------	--------	-------

Lower Limb

Beige	Black	Navy	Cocoa	Grey	Cranberry	Cherry
-------	-------	------	-------	------	-----------	--------

DAYTIME GARMENTS

JOBST® CUSTOM SEAMLESS SOFT

Custom-made Compression Hosiery

JOBST® Custom Seamless Soft is a custom-made circular knit garment that looks and feels like hosiery. Appropriate for mild to moderate edema and vascular disorders, these garments wick away moisture and help keep skin cool and dry. With Custom Seamless Soft, patients don't have to compromise on fit or style to wear medical grade compression.


KEY FEATURES

- Multi-fiber double covered inlay yarns increase softness and improve donning
- Refined opacity covers blemishes and scars
- High stretch core yarns increase recoil (stretch and recovery) to prevent sliding
- Reinforced foot zones increase strength and durability with added comfort


COLOURS



STYLES

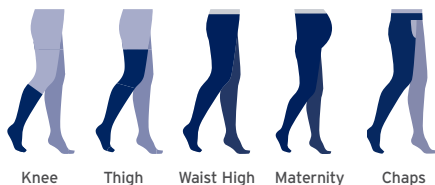

COMPRESSION CLASSES

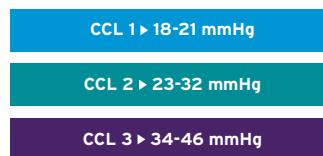

JOBST® BELLAVAR®

Firm Circular Knit Containmentment

JOBST® Bellavar® is designed to manage moderate to severe edema in the legs by providing controlled stretch and resistance. A custom-made circular knit garment, Bellavar® is seamless and provides a greater level of containment than JOBST® Custom Seamless Soft.


KEY FEATURES

- Designed to reduce wrinkling behind the knee
- Variety of styles and options to satisfy patient needs


COLOURS



STYLES

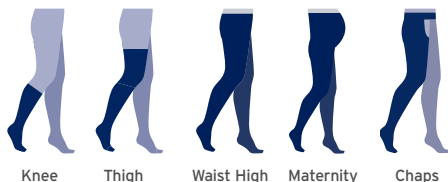

COMPRESSION CLASSES

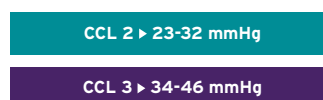

DAYTIME GARMENTS

JOBST® BELLA™ LITE

The Entry Point to JOBST® Quality

The JOBST® Bella™ Lite Ready-to-Wear Armsleeve and Gauntlet help maintain the limb size of lymphedema patients. These comfortable garments provide gradient compression to assist in patient management following trauma, surgery or radiation therapy.



KEY FEATURES


- Fits a wide range of arm and hand shapes and available with or without a silicone band
- Two armsleeve lengths for optimal fit
- Fine seams with ultra thin and smooth compression fabric

COLOURS

Beige

Black**

STYLES

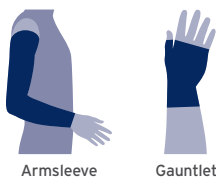

Armsleeve

Gauntlet

COMPRESSION CLASSES

15-20 mmHg*

20-30 mmHg*

*All compressions shown are the mean compression for an average forearm size (armsleeve) or mean compression for an average palm size (gauntlet). Not all colours and styles may be available in all compression ranges.

**Available in select styles.

JOBST® BELLA™ STRONG

Comfortable Compression

JOBST® Bella™ Strong upper extremity garment are the ready-to-wear solution for patients who need to manage their lymphedema, but do not need the containment provided by a flat-knit custom garment. Bella™ Strong makes JOBST® quality available to a large number of patients by offering a wide range of sizes.

KEY FEATURES

- Covers wide range of patient dimensions
- Easy to don, armsleeve comes with donning device
- Comfort Flat wrist seam reduces rolling
- Cotton-like feel

COLOUR

Beige

COMPRESSION CLASSE

20-30 mmHg*



STYLE



JOBST® Bella™ Strong Armsleeve

- Comfort Flat wristband reduces rolling and improves durability
- Available with silicone band
- Easy to don, armsleeve comes with donning device
- Available with knitted top band

*All compressions shown are the mean compression for an average forearm size (armsleeve)

DAYTIME GARMENTS

JOBST® CUSTOM SEAMED

Nearly Limitless Solutions for Unique Challenges

JOBST® Custom Seamed custom-made garments were the first of their kind to manage lymphedema, and while much has changed in the past 71 years, our ability to find solutions to unique challenges has not. Custom Seamed garments are available for every part of the body and in countless configurations. Every garment is precisely cut and sewn by hand.

KEY FEATURES

- Durable fabric for long lasting wear
- Precise gradient compression based on computer design
- Variety of options: zippers, linings, silicone bands and more

COLOURS

Beige

Black

COMPRESSION CLASS

15-65+ mmHg

STYLES



Mitten to Wrist



Mitten to Elbow



Gauntlet



Glove to Wrist



Armsleeve
(Glove to Elbow)



Elbow Band



Armsleeve
(Wrist to Axilla)



Armsleeve
with Attached
gauntlet



Face Mask



Chin Strap



Sleeveless Vest



Long-sleeved Vest



Short-sleeved Vest



Torso Band



Body Brief
with Sleeves



Sleeveless
Body Suit with
Legs



Panty Girdle
(Above Knee,
Closed Pubis)



Panty Girdle
(Below Knee,
Closed Pubis)



Abdominal
Pelvic
Garment



Foot Glove
(to Ankle)



Ankle
Garment



Knee High



Thigh
Band



Thigh High



Waist High



Stump
Garment



Waist High
Stump
Garment



DAYTIME GARMENTS

JOBST® SOFTFIT™

Our innovative JOBST® SoftFit™ technology is a unique system of silicone yarns knitted into the welt of our compression garments. It provides gentle, seamless friction on the skin, decreasing the chance of constriction and slipping down.

LOOKS GOOD, FEELS GOOD

With the SoftFit™ band, garments are not only visually appealing and more comfortable to wear, they provide precisely the right amount of pressure, right where it is needed.

IMPROVED COMPLIANCE, IMPROVED QUALITY OF LIFE

Compression garments that feel comfortable lead to greater patient compliance.

MORE OPTIONS, HAPPIER PATIENTS

SoftFit™ Technology is offered in a wide variety of compression garments and styles.



BAND VARIATIONS

Product	Dotted Silicone	Lace Silicone	SoftFit	Sensitive	Ribbed Fleece Waistband, Adjustable Waistband	MicroDot	Floral Waistband
Elvarex® Upper Extremity	2.5 cm, on top, inside 5 cm, on top, inside		3 cm, on top (CCL 1 and 2 only)			5cm, on top	
Elvarex® Lower Extremity	2.5 cm, on top, inside 5 cm on top, inside 5 cm on top, waistband		5 cm, on top (Knee High only, CCL 1 thru 3)	5 cm, on top (Waist High only)	5cm, on top (Waist High only)	5cm, on top	
Elvarex® Soft Upper Extremity	2.5 cm, on top, inside 5cm, on top only		3 cm, on top			5cm, on top	
Elvarex® Soft Lower Extremity	2.5 cm, on top (Knee High only) 5 cm, on top		5 cm, on top (Knee High only)		(Waist High only)	5cm, on top	
Confidence	5 cm, on top (Thigh High only) 2.5 cm on top for Armsleeve		5 cm, on top (Knee High only) 2.5 cm on top for Armsleeve				(Waist High only)
Bella™ Lite Armsleeve	5 cm, on top						
Bella™ Strong Armsleeve	5 cm, on top						
Custom Bellavar	5 cm, on top (Knee & Thigh High) 2.5 cm, on top (Knee High only)	6 cm, on top (Thigh High only)					
Custom Seamless Soft	5 cm, on top (Knee & Thigh High) 2.5 cm, on top (Knee High only)	6 cm, on top (Thigh High only)	5 cm, on top, (Knee High only) (CCL 1 and 2 only)	5 cm, on top, (Thigh High only)			
Custom Seamed	2.5 cm, on top 5 cm, on top			5 cm, on top			

FUNCTIONAL ZONES FOR JOBST® ELVAREX®, ELVAREX® SOFT & CONFIDENCE

Unique and innovative Functional Zones are created by the use of technologically advanced knitting techniques and specialized yarn selection. Functional Zones increase patient comfort through increased flexibility and breathability, without compromising medical effectiveness.

JOBST® FUNCTIONAL ZONES

- Use of highly specialized yarn and knitting techniques.
- More comfortable wearing due to increased breathability and flexibility.
- Special zones combat wrinkling when bending and speed moisture dissipation.



NIGHTTIME GARMENTS

JOBST® RELAX

Redefining Lymphedema Night Care

JOBST® Relax is the revolutionary nighttime garment that is designed for more than just sleeping. Its monofilament spacer fabric technology is unique in the marketplace and creates a supremely lightweight, flexible, and slim-line garment. Relax brings exceptional quality of life benefits and style to patients with mild to moderate lymphedema through the use of advanced knitting techniques and CoolMax® yarns. Because Relax is so breathable and flexible, patients can remain compliant through the evening hours.




KEY FEATURES

- Custom-made, flat-knit technology provides gradient compression therapy and stays in place
- Textured surface and monofilament spacer combine to create differential pressure areas that support stimulation of lymph flow while a patient is at rest

COLOURS




STYLES

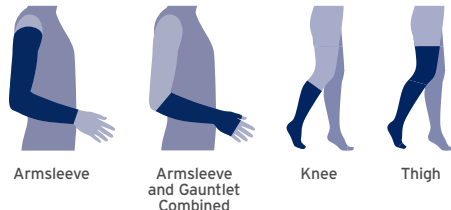

COMPRESSION CLASSES



* 20-30 mmHg compression is only available in lower extremity.

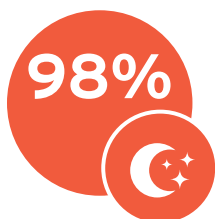


BETTER QUALITY OF SLEEP:

98% of patients rated quality of sleep **good or very good** compared to when wearing previous night-time products.



EDEMA CONTROL:

Many patients reported a major improvement in their lymphedema the next morning, with 90% of patients describing handling of day compression to be **good or very good** after using JOBST® Relax at night.

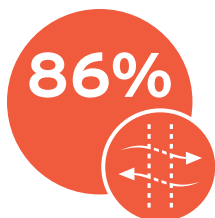

BREATHABILITY:

86.8% rated JOBST® Relax as **good or very good** in terms of avoiding heat generation and perspiration, respectively.



NIGHTTIME GARMENTS

JOBST® JOVIPAK®

Handmade Lymphedema Night Care

JOBST® JoViPak® garments are ideal for moderate to severe lymphedema patients who want an easy and comfortable alternative to nighttime bandaging. Unique continuous flow channeling and blended density chip foam construction allow JoViPak® garments to address even the most difficult edema and fibrotic tissue. Whether in ready-to-wear or custom configurations for every part of the body, each garment is handmade with care.

KEY FEATURES

- Variety of styles, sizes, and colours
- Custom and Ready-to-Wear
- Custom channeling options available
- Styles for virtually every part of the human anatomy
- Products to address any type of edema and fibrotic tissue
- Highly durable for increased longevity
- Uses up to four kinds of foam with no unpleasant chemical odours

COLOURS

Cotton



PowerNet



Super PowerNet



Polartec® Power Dry®



JOBST® JOVIPADS®

JOBST® JoViPads® can be used in any area where there is a need for softening fibrotic tissue, providing pressure equalization, filling an anatomical indentation, or adding protection between skin folds.



JOBST® PITPAKS

JOBST® PitPaks are filled with sterilized cherry pits and were specially designed to help address more difficult areas of fibrosis. These are an aggressive type of therapy and should only be undertaken with close therapist supervision.



JOVILINER ARM



Extends from the Distal Interphalangeal Joints (DIPs) to the axilla with stitching between each finger. Liners to be used under short stretch bandaging or pump appliances.

JOVILINER LEG



Full Leg has Continuous Flow Channels from the tip of the toes to the groin. Liners to be used under short stretch bandaging or pump appliances.

NIGHTTIME GARMENTS

BELLISSE®

Supporting Healing After Surgery

Bellisse® is uniquely designed to promote healing after breast surgery by providing uniform, gentle compression around the torso. When used in conjunction with JOBST® JoViPads®, Bellisse® encourages drainage of edema in the chest walls, which can aid in recovery.

KEY FEATURES

- Ready-to-Wear and available in a large range of sizes
- Unprecedented adjustability with both a front zipper for easy donning and a variable position back closure
- Attractive shape that can be worn comfortably day and night
- Soft to the skin, smooth while sleeping and unobtrusive under clothing
- Made of Polyester / Nylon / Spandex



Bellisse® is exclusively distributed by JOBST®

COLOURS

Buff

Black

SIZING RANGE

32 A/B - 50 DD/E*

*Please check order form with measuring instructions to determine the correct size.



BREAST AND CHEST WALL PADS

Breast and Chest Wall Pads are comfortable, foam-filled pads that conveniently fit into the Bellisse® to address specific areas of breast and chest wall fibrosis.



Cleavage



Chest Wall Pocket



Post-Lumpectomy



Padded Insert



Unilateral
Post-Mastectomy



Inframammary



Axilla



Mini-Axilla



Serratus Anterior



Double Mastectomy



Lateral

ACCESSORIES



JOBST® Donning Gloves*

Putting on JOBST® compression garments is always easier with JOBST® Donning Gloves! They help create friction, provide grip and help to evenly distribute the fabric on the leg.



JOBST® Stocking Donner

Easy to use device to help apply JOBST® compression garments. Ideal for people who have limited dexterity or have difficulty reaching their feet. Use with closed and open toe styles.



Dycem®

This 8" x 11" Dycem® is placed on the floor or table top to provide a non-slip surface the patient can push against as they don either an arm or leg garment. Dycem® can be easily cleaned with soap and water to renew its non-slip surface.



Arion Application Aids

Arion Application Aids assist in the donning and doffing of upper and lower extremity garments. These application aids can be used for both open ended and closed ended garments, and can be very helpful for patients with tenderness or weakness in their donning limb or are not strong enough to don the garment themselves.



JOBST® Roll-On Adhesive

- Designed to keep JOBST® Hosiery in place
- Ideal for thigh high lengths that do not have a silicone band or when a garter belt is not an option
- Helpful Hint: For flat-knit garments, apply in vertical strips

PREFER TO SHOP ONLINE?

JOBST® eShop

- Place your JOBST® Custom Lymphology orders online
- Intelligent patient management
- Easy measurement documentation
- Time efficient ordering
- Voice enabled order form documentation with JOBST® Xpert Voice



To get started, visit <https://eshop.jobst-ca.com>

Questions?

Email us at ca.customerservice@essity.com or call us at 1-877-978-5526.

Sorbact® is a registered trademark of ABIGO Medical AB.
Bellisse® is a registered trademark of Lightning2, LLC.
VELCRO® is a registered trademark of Velcro BVBA.
CoolMax® is a registered trademark of INVISTA.
Dycem® is a registered trademark of Dycem Ltd.
Easy-Slide® is a registered trademark of Arion International.

Please refer to the product label and / or package insert for full instructions on the safe use of these products.



JOBST®,
an Essity brand

 jobstcanada.com essity.com

Essity
1275 North Service Road West, Suite 800
Oakville, ON Canada L6M 3G4
Tel. 1-877-978-5526 Fax 1-877-978-9703
136342E ©Essity Hygiene and Health AB, 2025 J25