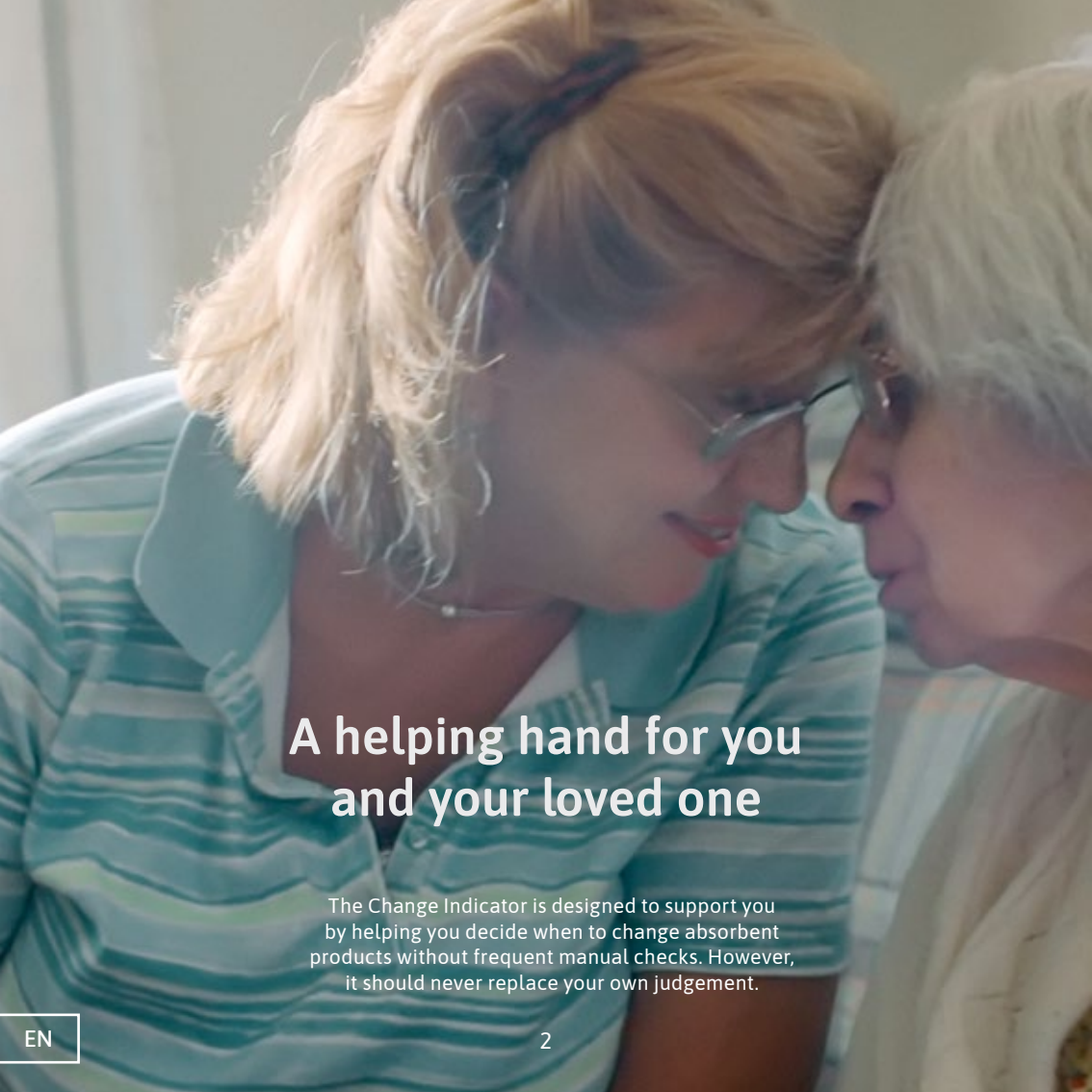
QUICK GUIDE

TENA SmartCare Change Indicator™

EN	For family caregivers	02
ES	Para cuidadores	16
PT	Para cuidadores familiares	30
FR	Pour les aidants familiaux	44
IT	Per i caregiver familiari	58
DE	Für pflegende Angehörige	72
NL	Voor mantelzorgers	86

A helping hand for you and your loved one

The Change Indicator is designed to support you by helping you decide when to change absorbent products without frequent manual checks. However, it should never replace your own judgement.

Important information

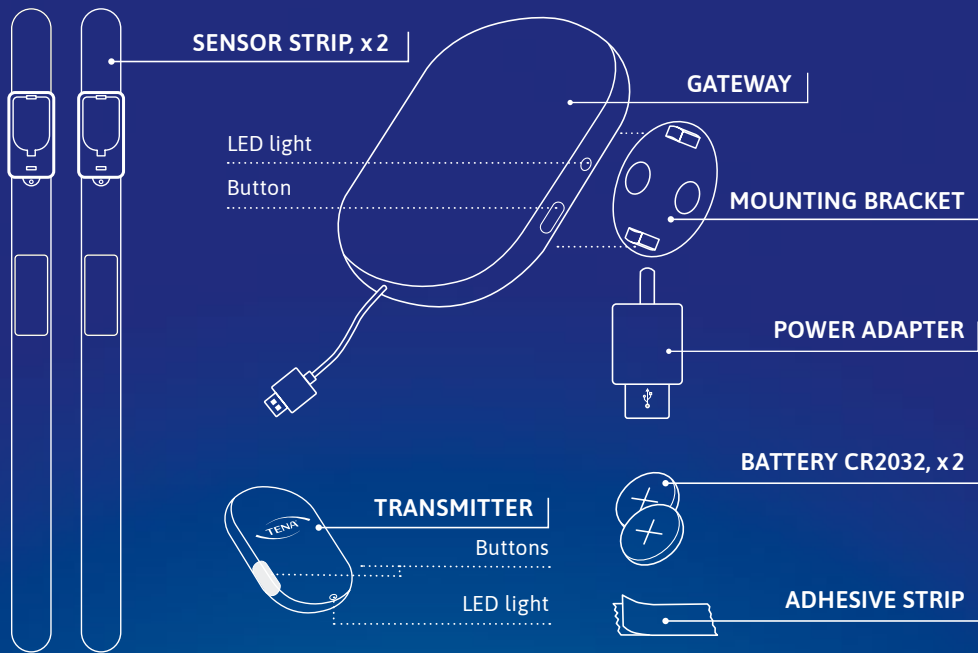
The Change Indicator is intended for use with TENA incontinence products.

Clean the Change Indicator when fully assembled to prevent water from entering the device. Do not submerge in water or clean in a washing machine.

Do not attempt to repair or modify the Change Indicator or a gateway by yourself.

Seek immediate medical care if the battery is swallowed.

What's in the box

You also need (not included):

**TENA
ABSORBENT
PRODUCT**

SMARTPHONE
iOS 11/Android 7
or later

1 Start with the app

1.1 DOWNLOAD APP AND REGISTER ACCOUNT

Scanning the relevant QR code will take you directly to the App Store or Google Play, where you can download the TENA SmartCare Family Care app.

To App Store (iOS)

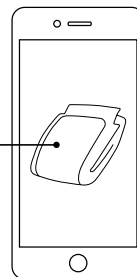
To Google Play (Android)

Once you have downloaded the app, start setting up the Change Indicator using the instruction films in the app or this quick guide.

1.2 REGISTER YOUR LOVED ONE'S PRODUCT IN THE APP

For the most accurate results, register the absorbent product your loved one is currently using in the app.

2 Set up the gateway

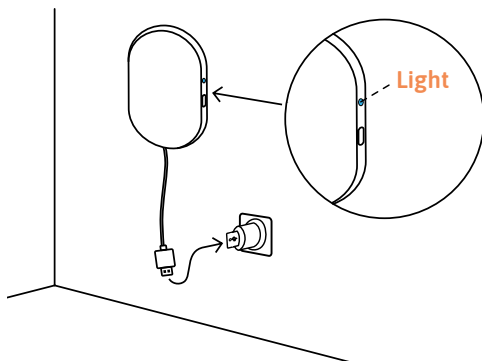
CONNECT


CONSIDER RANGE (33 FT/ 10 M)

The gateway can be mounted on a wall or placed on a flat surface.


2.1 Connect to power outlet

Blinking light
= connecting

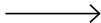
Steady light
= connected

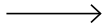No light
= OK

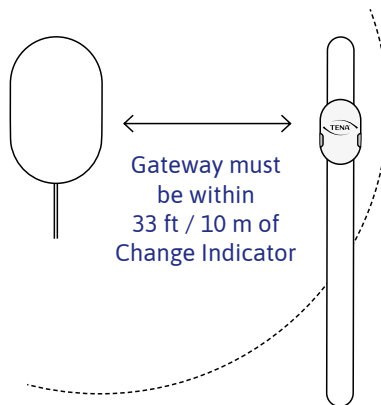2.2 Consider range


Allow up to 60 seconds for the light to start blinking and another 60 seconds for it to turn green.

3 Register the transmitter

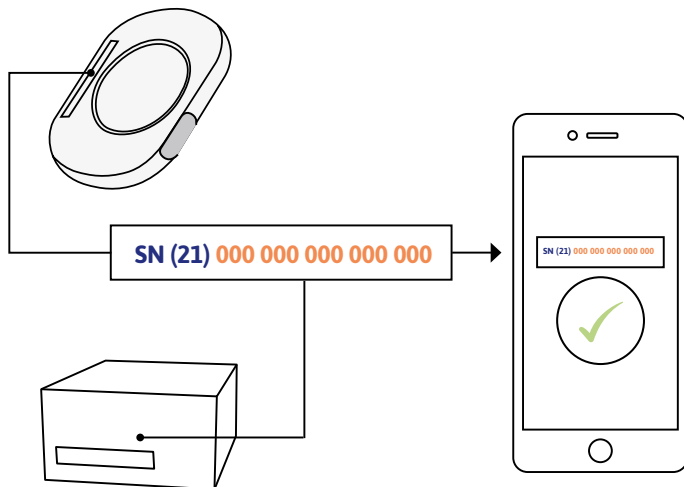On the back of the transmitter, and on the transmitter packaging, there is a 15-digit code. To register the transmitter, enter the 15-digit code in the app. The app will confirm when the code has been approved.


4 Assemble the Change Indicator


INSERT BATTERY IN THE TRANSMITTER

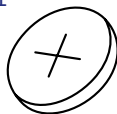4.1

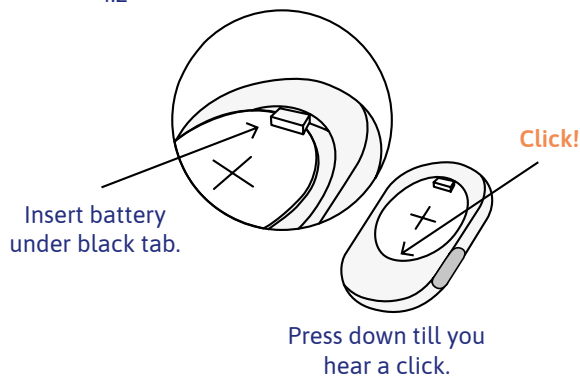
4.2




Press down till you hear a click.

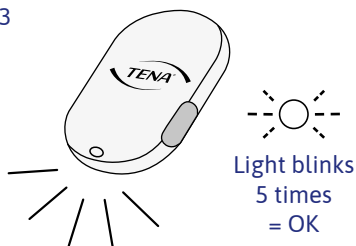4.3

ATTACH TRANSMITTER TO SENSOR STRIP

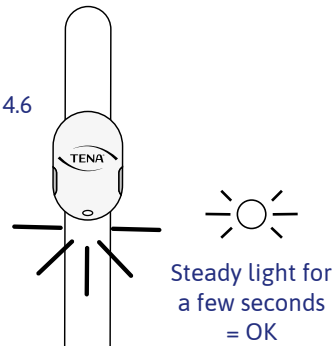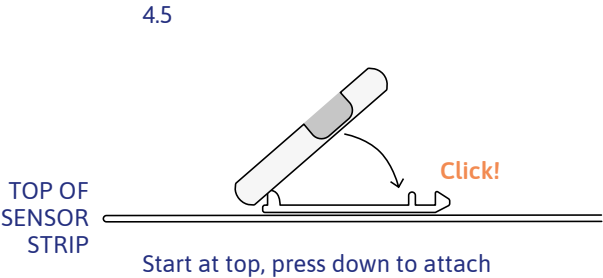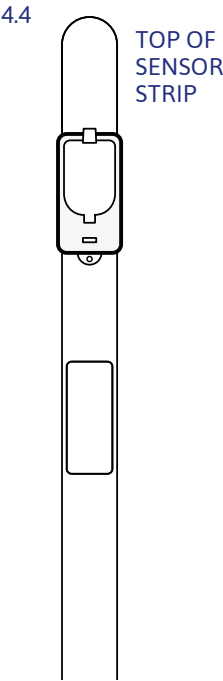5 Attach Change Indicator to product

FIND THE FRONT

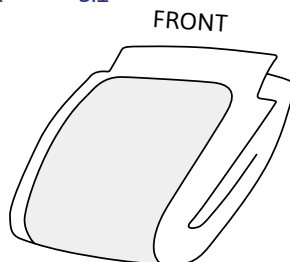
KEEP FOLDED


TENA Slip/TENA Flex

5.1

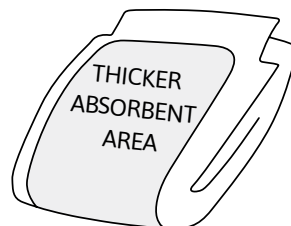
5.2


TENA Pants

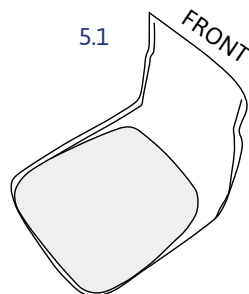5.1

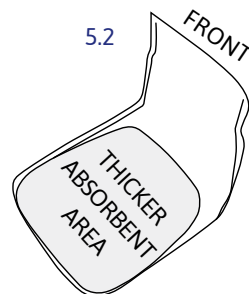
5.2



ALIGN WITH THE TOP OF THE
THICKER ABSORBENT AREA

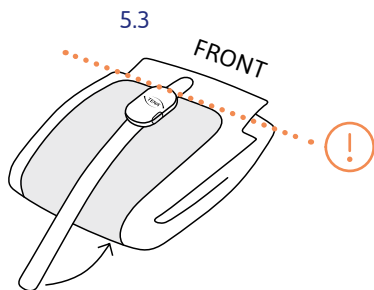
PLACE TIGHTLY

Press firmly! Tight fit all the way

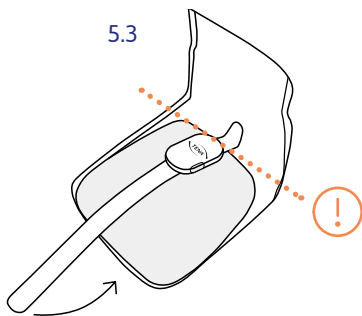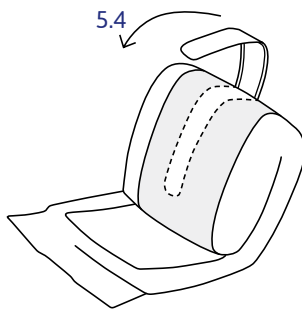Press firmly! Tight fit all the way

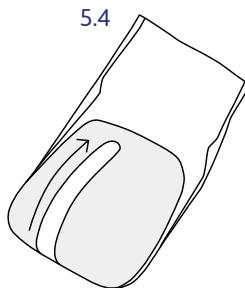
6 Use the Change Indicator

PRESS BUTTONS

APPLY THE TENA PRODUCT

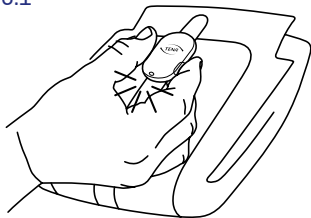6.1


Press both buttons
for 2 seconds.

Steady light for a
few seconds = OK

Press both buttons simultaneously
for two seconds after every change.
This registers the change and
resets the system.

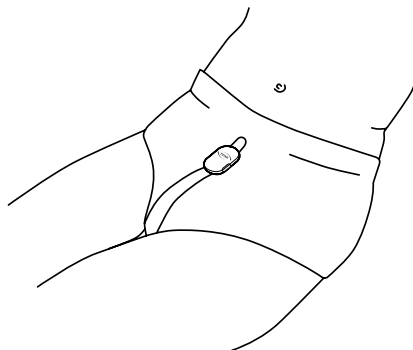6.2


Apply on your loved one.

If the user is very mobile,
consider applying a fixation pant
or ordinary underwear on top of
the Change Indicator. This will
keep it comfortably in place.

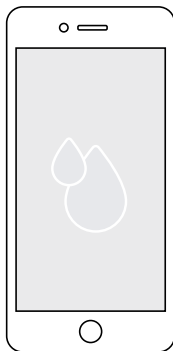6.3

**DRY****– no action needed**

The absorbent part of the TENA product is dry.

6.4

**SOME URINE****– no action needed**

Some urine in the absorbent part of the TENA product. The surface still feels dry to the skin.

6.5

**WET****– consider change**

More urine in the absorbent part of the TENA product. It probably still feels dry to the skin, but you should consider change.

In case of error notifications, consult the instructions for use for more information.

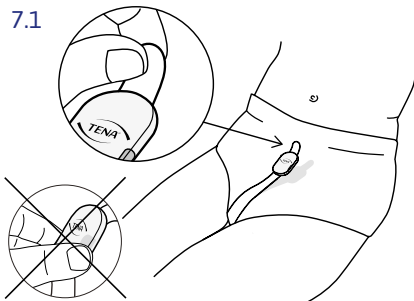
7 Remove and clean

REMOVE THE CHANGE INDICATOR

CLEAN WHEN ASSEMBLED

7.1

Remove Change Indicator before removing the TENA product.

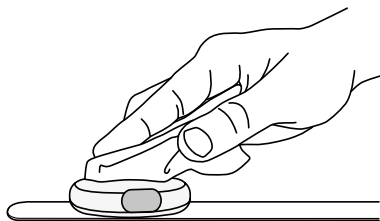
Start at the top. Grip the tab, then gently peel straight downwards.

Place Change Indicator on a flat surface when not in use.

Remember to press both buttons simultaneously for two seconds before or after every change. This registers the change and resets the system.


7.2

Only clean when assembled.

Clean once a week or as needed.

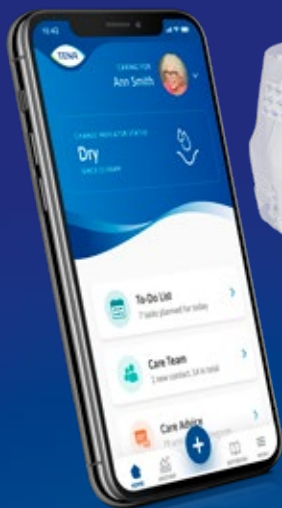Use an alcohol-based disinfectant when needed.

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