

A photograph of two women in a living room. The woman in the foreground is seen from the back, wearing a red t-shirt and dark blue shorts with a light green stripe. She is wearing beige compression leggings and has her right arm raised. The woman in the background is also wearing a red t-shirt and similar shorts, and is captured in a jumping pose with both arms raised. She is also wearing beige compression leggings. The room has a brick wall, a large potted plant, a brown sofa with orange cushions, and a wooden pendant lamp. A blackboard with photos is visible in the background.

COMPRESSION THAT STANDS UP TO LYMPHEDEMA

DEFY GRAVITY

JOBST

Help your patients stand up to lymphedema with
a **complete range of compression products** designed
with them in mind.

JOBST® Educational Resources

The JOBST® Compression Institute serves as a resource for healthcare providers and clinicians seeking further education about vein diseases and lymphedema. This website provides literature, videos, and links to our national affiliates. Additionally, the JOBST® Compression Institute offers a variety of nationwide workshops and continuing education classes to train you on JOBST® products and proper measuring and fitting techniques.

Register at <http://www.jobstcompressioninstitute.com>.


To connect with a JOBST® Medical Advisor, please call our Customer Service team at **1-800-537-1063**.



PREFER TO SHOP ONLINE?

JOBST® eShop

- Place your JOBST® Custom Lymphology orders online
- Intelligent patient management
- Easy measurement documentation
- Time efficient ordering
- Voice enabled order form documentation with JOBST® Xpert Voice

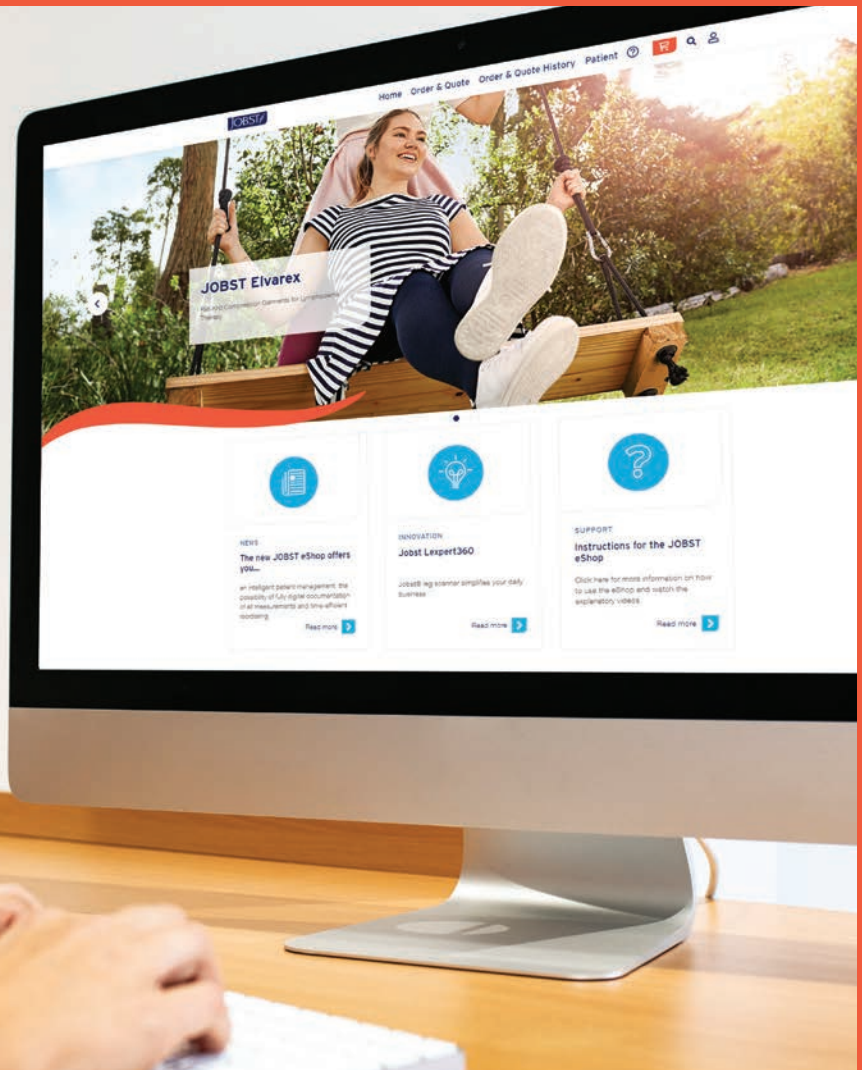

To get started, visit <https://eshop.jobst-usa.com>

Questions?

Email us at HMS-Elvarex-Orders@essity.com or call us at **800-537-1063**.

THE COMPLETE SOLUTION FOR LYMPHEDEMA

You are passionate about finding the right products for your patients, as you don't want them to be held down by their condition. JOBST® offers a complete portfolio of products to support you and your patients.

JOBST® Educational Resources / Custom eShop	2	JOBST® Custom Seamed	15
Why JOBST®?	4	JOBST® SoftFit™	16
Compression Bandages		Band Variations.....	16
JOBST® Compri2.....	5	Functional Zones for JOBST® Elvarex®,	
JOBST® FarrowWrap®	6-7	JOBST® Elvarex® Soft and JOBST® Confidence.....	17
JOBST® Daytime Garments Decision Tree.....	8	JOBST® Relax	18
JOBST® Elvarex®	9	JOBST® JoViPak®	19-20
JOBST® Elvarex® Soft.....	10	Bellisse®	
JOBST® Confidence.....	11	Breast and Chest Wall Pads.....	21
JOBST® Elvarex® Plus		JOBST® JoViPads®	
JOBST® Elvarex® Soft Seamless	12	JOBST® JoViLiner Arm and Leg	
JOBST® Custom Seamless Soft		JOBST® PitPaks.....	22
JOBST® Bellavar®	13	JOBST® Donning Aids & Accessories	23
JOBST® Bella™ Lite			
JOBST® Bella™ Strong	14		



WHY JOBST®?



TRUSTED BRAND

- JOBST® is the #1 physician-recommended brand of medical compression ready-to-wear legwear in the US* for the prevention, treatment and management of venous and lymphatic conditions.
- Conrad Jobst, mechanical engineer and industry pioneer, dedicated his life and expertise to inventing graduated compression garments in 1950.

INNOVATION

- JOBST® incorporates customer and healthcare professional involvement in the development of products. From focus groups to wash-and-wear tests, all garments go through a rigorous design process that ensures they provide the highest efficacy, safety, and satisfaction for our customers.
- Innovation is at the heart of our product excellence. Product development efforts are directed towards delivering the highest quality product with the highest assurance of medical efficacy.



PARTNERSHIPS

- JOBST® is an exclusive sponsor of the JOBST® Research Grant in Venous and Lymphatic Diseases in association with the American Venous Forum Foundation.
- JOBST® supports the National Lymphedema Network (NLN), Lymphatic Education and Research Network (LE&RN), and Bryan's Feat Foundation.
- JOBST® sponsors the JOBST® Lymphatic Research Grant through the Boston Lymphatic Symposium supporting research in lymphatics and lymphatic disease.
- Key JOBST® stakeholders are part of the Lymphedema Advocacy Group. Their mission is to advocate for improved insurance coverage for the diagnosis and treatment of the disease.

EDUCATION

- JOBST® Compression Institute serves as a valuable resource for fitters, clinicians, healthcare providers and caregivers interested in learning more about compression therapy and lymphatic and venous disorders.
- Our product experts, called Medical Advisors, and our devoted team of Sales Professionals, provide customized education and solutions for both fitters and therapists.



SOCIAL RESPONSIBILITY



- Every year, JOBST® donates hundreds of compression garments to healthcare organizations who share our mission to help improve patient quality of life. Whether it is funding education through organizations such as Camp Watch Me, the NLN Garment Fund, or the LE&RN, or helping those recover after a devastating natural disaster, JOBST® does not hesitate to lend a helping hand.

COMPRESSION BANDAGES

A complete range of bandage products for the management of lymphedema. Designed with the patient and therapist in mind for optimal Complete Decongestive Therapy (CDT) outcomes.

MULTI-LAYER BANDAGING



Comprilan®

- Short stretch compression bandage
- Provides high working pressure and low resting pressure
 - Easy care and durable



Elastomull®

- Soft comfortable bandage
- Precision fit in areas with constant movement
 - Air-permeable and comfortable to wear



Tricofix®

- Lightweight stockinette
- Protects the skin surface under compression, casts, or lymphedema wrapping
 - 100% cotton



CompriFoam®

- Open cell foam bandage
- 100% polyurethane foam
 - Reusable



Isoband®

- Elastic multipurpose bandage
- Short stretch with longitudinal elasticity for high working pressure
 - Reusable
 - Porous weave provides good ventilation to the skin



Artiflex®

- Non-woven padding bandage
- Used in Lymphology bandaging
 - Provides padding around bony or irregular shapes
 - Conformable and air-permeable
 - Supports even distribution of pressure

JOBST® COMPRI2 2-Layer Bandaging

The JOBST® Compri2 is a 2-Layer Bandage System* specifically designed for easy application. It provides graduated and reliable compression for up to seven days to help achieve effective edema reduction and helps promote wound healing.



1st Layer - Non Woven Padding

- **Stretchable** - simplifies application
- **Active Moisture Management** - helps keep skin dry and comfortable
- **Customized Padding** - allows for anatomical adjustments and additional protection to sensitive areas



2nd Layer - Short Stretch Bandage

- **Cohesive** - delivers reliable compression
- **Adjustable** - allows modification during application
- **Visual** - indicator offers simple and effective application support

*As JOBST® Compri2 does not include a wound dressing, any wounds must be treated with a suitable wound dressing before the compression bandage is applied.

COMPRESSION WRAPS

JOBST® FARROWWRAP®


A guide to selecting the right JOBST® FarrowWrap®

APPLICATION TIME:

**43%
FASTER**

using JOBST® FarrowWrap®
compared to 4-layer
compression bandages.

COMPRESSION LEVEL

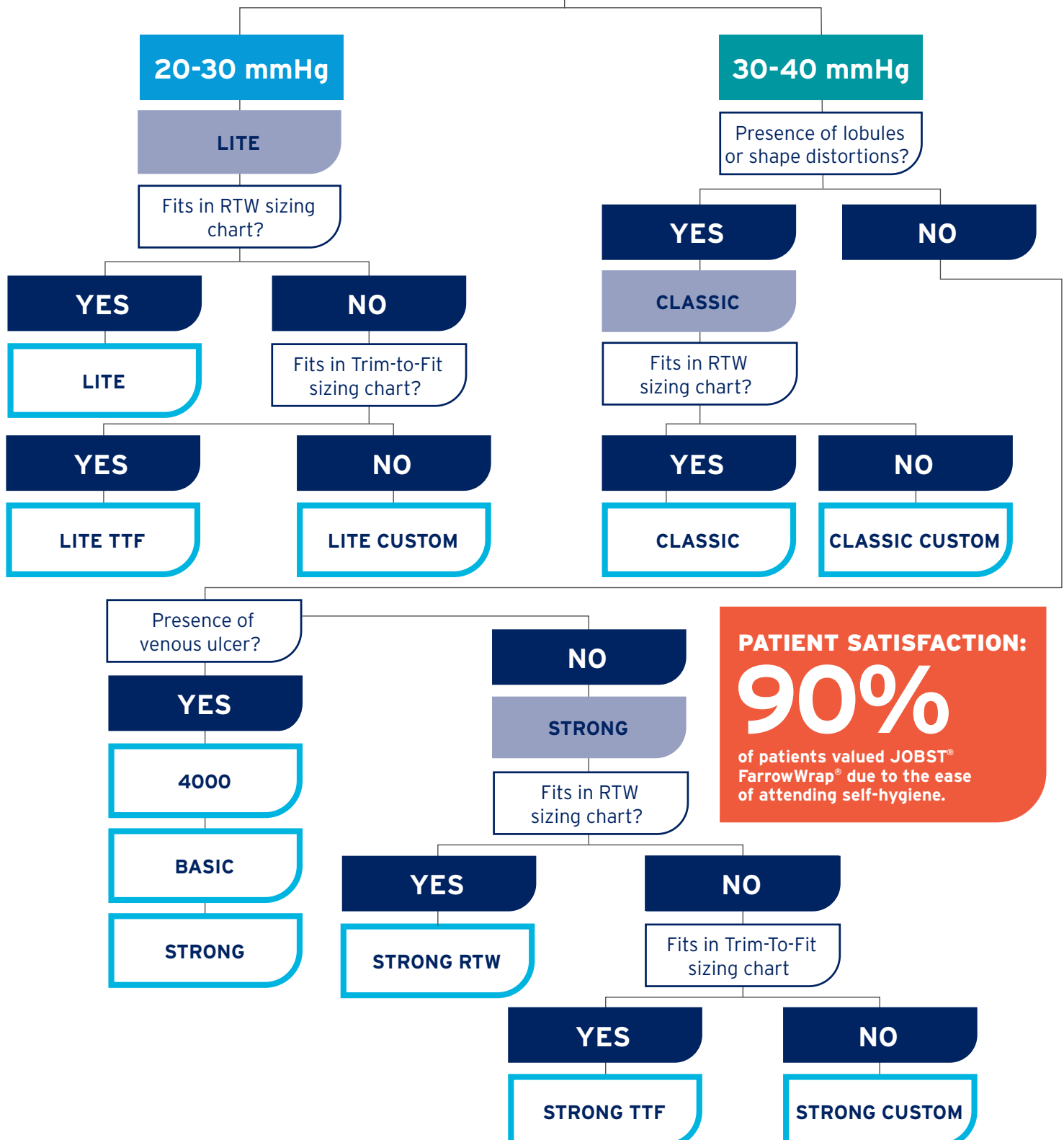

PATIENT SATISFACTION:

90%

of patients valued JOBST®
FarrowWrap® due to the ease
of attending self-hygiene.

COMPRESSION WRAPS

JOBST® FARROWWRAP®


A FarrowWrap® style to suit every need

JOBST® FarrowWrap® STRONG (30-40 mmHg*)

Premium fabric with strong compression and a soft, inner layer suitable for patients with good limb shape. Recommended for patients with moderate to severe edema. Available in Trim-to-Fit.



Footpiece^{1,2,3}

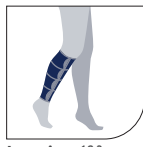


Legpiece^{1,2,3}



Kneepiece¹



Thighpiece^{1,2,4}

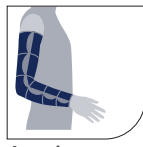

Armpiece (Custom)

Colors

Tan


Black

JOBST® FarrowWrap® CLASSIC (30-40 mmHg*)

Made from more rigid material than JOBST® FarrowWrap® STRONG. Ideal for those with large lobules, skin folds or shape distortions.



Footpiece^{1,2}

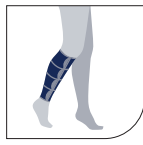


Legpiece^{1,2}



Kneepiece^{1,5}



Thighpiece^{1,2,4}

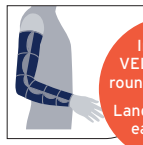

Armpiece² (Custom)

Colors

Tan


Improved VELCRO® with rounded corners. Landing strip at each strap.

JOBST® FarrowWrap® BASIC (30-40 mmHg*)

A value fabric with quality compression. The double-sided VELCRO® enables the band length to be trimmed as the limb reduces.



Footpiece¹

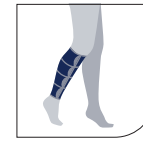


Legpiece¹ (Includes FarrowHybrid Sock)

Colors

Tan

JOBST® FarrowWrap® 4000 (30-40 mmHg*)

JOBST® FarrowWrap® 4000 is ideal for patients with moderate to severe venous and lymphatic conditions with or without a venous leg ulcer. The inner sleeve and 4-band juxtaposing design make for easy application. Anatomically contoured to help prevent gapping.

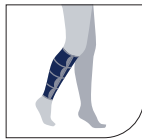

Legpiece¹ (Includes FarrowHybrid Sock)

Colors

Tan


Black

JOBST® FarrowWrap® LITE (20-30 mmHg*)

A double-laminated, durable and breathable fabric especially suitable for home care use/self-management. Available in Trim-to-Fit.



Footpiece^{1,2,3}

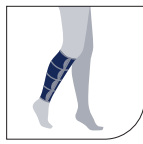


Legpiece^{1,2,3}



Kneepiece^{1,5}



Thighpiece^{1,2,4}

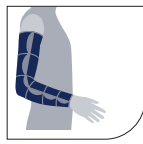

Armpiece^{1,2,3}

Colors

Tan

Black



Hand Gauntlet**

The JOBST® FarrowWrap® Hand Gauntlet manages swelling in the hand and can be used on either the left or right hand. Includes removable 4 mm and 8 mm hook and loop fasteners.



Toe Cap** (15-20 mmHg*, 20-30 mmHg*)

Trimable Ready-To-Wear toe cap for managing swelling in the forefoot and toes. Can be used on either foot, and is available in black and tan.



FarrowHybrid ADI Foot Compression Sock

Below knee high liner with 20-30 mmHg of compression in the foot and ankle area. Included with BASIC and 4000 Legpiece.

*The average compression for a medium sized ankle.

**Hand Gauntlet and trimmable RTW Toe Cap for managing mild to moderate edema.

1. RTW (Ready-To-Wear) 2. CM (Custom Made) 3. TTF (Trim-To-Fit) 4. Thighpiece must be worn with Kneepiece 5. STRONG kneepiece can be combined with CLASSIC garment



Easy Donning

Easy-to-attach VELCRO® brand tabs support easy donning and doffing.



Short-Stretch Technology

Short-Stretch Technology provides low-resting and high-working pressures to enhance lymphatic and venous return.



Adjustable

VELCRO® brand hook & loop system allows patients to respond better to fluctuating edema by simply tightening or loosening the wrap.



50% Band Overlap

Optimal width of bands - combined with 50% overlap of bands from bottom to top - provide ideal compression.



Liner Options

- Standard Liner
- TG Soft Liner
- FarrowHybrid Liner

Additional Accessories

- Garmet Grip
- VELCRO® Pack
- Short-Strech Wrap Extra Band

DAYTIME GARMENTS

FLAT-KNIT DECISION TABLE:

JOBST® Confidence vs JOBST® Elvarex® Soft vs JOBST® Elvarex®

Body Shape			
Stage	Mild (I)	Moderate (II)	Severe (III)
Patient Description	First time user, newly diagnosed, preventative treatment, Lipedema/ Lymphedema	First time user, low tolerance to stiff fabric, desire to “feel” the compression, Lipedema/ Lymphedema	Persistent/aggressive edema, Lymphedema (not best for Lipedema)
Skin Folds	None	Minimal	Significant
Tissue	Proportionate, symmetrical, athletic, controlled/maintained edema	Adipose present, beginning development of fibrosis	Significant adipose and/or hard, fibrotic
User Need	Light & Soft material	Soft & Flexible material	Firm & Strong material
JOBST® Product Recommendation	JOBST® Confidence		JOBST® Elvarex®
		JOBST® Elvarex® Soft	
			JOBST® Elvarex®

DAYTIME GARMENTS

JOBST® ELVAREX®

JOBST's Gold Standard

JOBST® Elvarex® represents JOBST's gold standard in custom daytime garments for the management of lymphedema. These flat-knit garments offer the very best in comfort and containment for even the most severe cases. With a natural rubber core*, Elvarex® provides unsurpassed structure and control for patients of all shapes and sizes.

KEY FEATURES

- Flat-knit structure bridges skin folds instead of working into them
- Natural rubber latex core provides superior containment and allows garment to return to original shape for extended wear
- Excellent breathability for comfort and patient compliance
- Seven compression classes for optimal therapeutic outcomes
- Vast product assortment covers a diverse range of specifications and requirements of patients
- SPF 80 UV protection



EXTREMITIES/COLORS


Upper Limb

Beige	Black	Navy	Caramel	Cranberry	Cherry	Hazelnut	Bronze
-------	-------	------	---------	-----------	--------	----------	--------

Lower Limb

Beige	Black	Navy	Cocoa	Cranberry	Gray	Henna**	Graphite**
Denim**	Stone	Aubergine	Caramel	Bronze			

STYLES

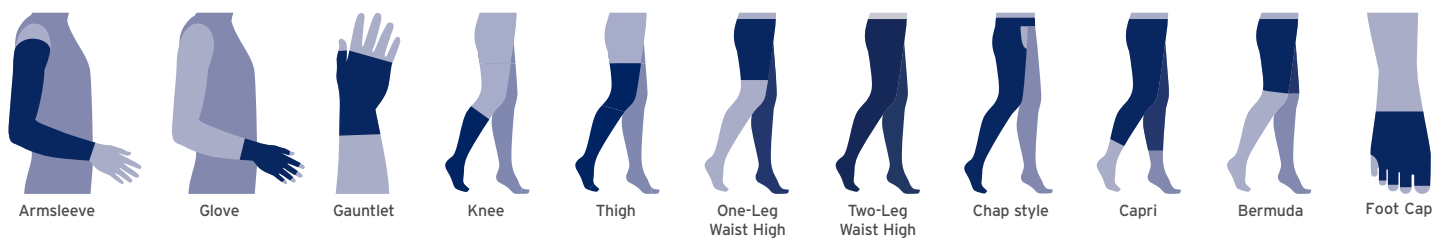

COMPRESSION CLASSES

Upper Limb

CCL 1 ▶ 15-21 mmHg
CCL 2 ▶ 23-32 mmHg
CCL 2 Forte ▶ 23-32 mmHg

Lower Limb

CCL 1 ▶ 18-21 mmHg	CCL 3 Forte ▶ 34-46 mmHg
CCL 2 ▶ 23-32 mmHg	CCL 4 ▶ 49-70 mmHg
CCL 3 ▶ 34-46 mmHg	CCL 4 Super ▶ 60-90 mmHg



*CAUTION: These products contain natural rubber latex which may cause allergic reactions.

**Not available in Foot Cap

DAYTIME GARMENTS

JOBST® ELVAREX® SOFT

Gentle and Not Made with Natural Rubber Latex

JOBST® Elvarex® Soft is the solution for patients wanting a softer, gentler compression garment. The flat-knit fabric allows for easier donning and meets the specific needs of many elderly and pediatric patients, or patients with lipedema. While soft and forgiving, Elvarex® Soft still provides the containment that mild to moderate lymphedema patients need.



EXTREMITIES/COLORS


Upper Limb

Beige	Black	Navy	Cocoa	Gray	Cranberry	Cherry	Honey
-------	-------	------	-------	------	-----------	--------	-------

Lower Limb

Beige	Black	Navy	Cocoa	Gray	Cranberry	Cherry
-------	-------	------	-------	------	-----------	--------

STYLES

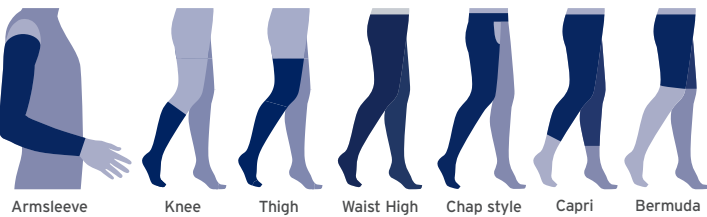

COMPRESSION CLASSES

Upper Limb

CCL 1 ▶ 15-21 mmHg
CCL 2 ▶ 23-32 mmHg

Lower Limb

CCL 1 ▶ 18-21 mmHg
CCL 2 ▶ 23-32 mmHg
CCL 3 ▶ 34-46 mmHg



DAYTIME GARMENTS

JOBST® CONFIDENCE

Next Generation of Made-to-measure Flat-knit Compression Garments


JOBST® Confidence helps your patients feel unrestricted, with a one-of-a-kind fit garment that's as unique as they are. The new generation of flat-knit technology used in JOBST® Confidence allows for a precise fit to the real anatomical body shapes. This personally refined fit not only provides comfort, but also supports your patients with a greater degree of movement.

KEY FEATURES

- 3D innovative knitting method allows the garment to conform to individual body shapes
- Flexible, conforming fabric makes it easy to put on and take off, while providing firm comfort
- Moisture wicking with double-layer construction for optimal comfort
- Not made with natural rubber latex



COLORS

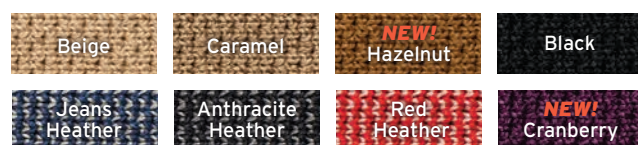

MOTIVATIONAL WAISTBAND



SEAMS



DESIGN OPTIONS



COMFORT OPTIONS



STYLES



COMPRESSION CLASSES

Upper Limb	Lower Limb
CCL 1 ▶ 15-21 mmHg	CCL 1 ▶ 18-21 mmHg
CCL 2 ▶ 23-32 mmHg	CCL 2 ▶ 23-32 mmHg
	CCL 3 ▶ 34-46 mmHg

PATIENTS PREFER THE COMFORT OF JOBST® CONFIDENCE

RANGE OF MOTION:

85% of patients said they can move freely with JOBST® Confidence.

MOISTURE MANAGEMENT:

92% of patients agreed JOBST® Confidence keeps the skin dry.

GENERAL WEARING COMFORT:

84% of patients evaluated the comfort of JOBST® Confidence as good/very good.

PATIENT SATISFACTION:

7/10 patients said they would recommend JOBST® Confidence.

DAYTIME GARMENTS

JOBST® ELVAREX® PLUS*

The Cutting Edge of Flat-Knit Technology

JOBST® Elvarex® Plus uses advanced flat-knitting technology to construct garments in three dimensions to more closely conform to the shape of patients' hands and feet. These seamless garments help minimize potential irritation between the fingers and toes, and at the base of the thumb, help patients with the comfort they want and the performance they need.

*Caution: This product contains natural rubber latex which may cause allergic reactions.



EXTREMITIES/COLORS


Upper Limb

Beige	Black	Navy	Caramel	Cranberry	Red	Hazelnut	Cherry	Bronze
-------	-------	------	---------	-----------	-----	----------	--------	--------

Lower Limb

Beige	Black	Navy	Caramel	Cranberry	Red	Gray	Cocoa	Bronze
-------	-------	------	---------	-----------	-----	------	-------	--------

STYLES

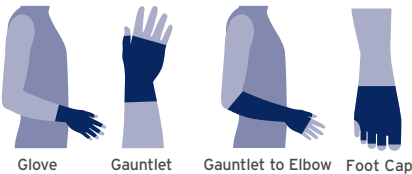

COMPRESSION CLASSES

CCL 1 ▶ 15-21 mmHg (Glove/Gauntlet)
CCL 1 ▶ 18-21 mmHg (Foot Cap)
CCL 2 ▶ 23-32 mmHg



JOBST® ELVAREX® SOFT SEAMLESS

Soft and Seamless, Where it Matters Most


JOBST® Elvarex® Soft Seamless combines all of the benefits of Elvarex® Soft with the seamless construction that patients want in a glove or toe cap. Not only are these flat-knit garments soft and not made with natural rubber latex, they are seamless, improving comfort by helping to eliminate potential irritation between the fingers and toes.



KEY FEATURES

- Soft, durable, and breathable flat-knitted fabric
- Flat edge finish at fingers and toes decreases bulk and prevents curling
- Easy to don, comfortable to wear

STYLES

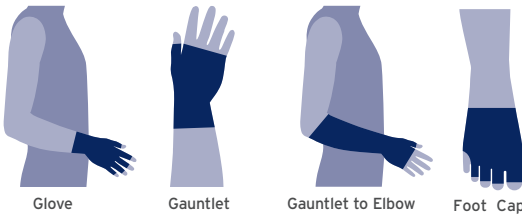

COMPRESSION CLASSES

CCL 1 ▶ 15-21 mmHg (Glove/Gauntlet)
CCL 1 ▶ 18-21 mmHg (Foot Cap)
CCL 2 ▶ 23-32 mmHg

EXTREMITIES/COLORS

Upper Limb

Beige	Black	Navy	Cocoa	Gray	Cranberry	Cherry	Honey
-------	-------	------	-------	------	-----------	--------	-------

Lower Limb

Beige	Black	Navy	Cocoa	Gray	Cranberry	Cherry
-------	-------	------	-------	------	-----------	--------

DAYTIME GARMENTS

JOBST® CUSTOM SEAMLESS SOFT

Custom-made Compression Hosiery

JOBST® Custom Seamless Soft is a custom-made circular knit garment that looks and feels like hosiery. Appropriate for mild to moderate edema and vascular disorders, these garments wick away moisture and help keep skin cool and dry. With Custom Seamless Soft, patients don't have to compromise on fit or style to wear medical grade compression.

KEY FEATURES

- Multi-fiber double covered inlay yarns increase softness and improve donning
- Refined opacity covers blemishes and scars
- High stretch core yarns increase recoil (stretch and recovery) to prevent sliding
- Reinforced foot zones increase strength and durability with added comfort

COLORS



STYLES



COMPRESSION CLASSES



JOBST® BELLAVAR®

Firm Circular Knit Containment

JOBST® Bellavar® is designed to manage moderate to severe edema in the legs by providing controlled stretch and resistance. A custom-made circular knit garment, Bellavar® is seamless and provides a greater level of containment than JOBST® Custom Seamless Soft.

KEY FEATURES

- Designed to reduce wrinkling behind the knee
- Variety of styles and options to satisfy patient needs


COLORS



STYLES



COMPRESSION CLASSES

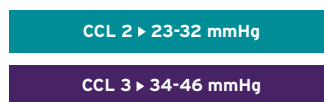

DAYTIME GARMENTS

JOBST® BELLA™ LITE

The Entry Point to JOBST® Quality

The JOBST® Bella™ Lite Ready-to-Wear Armsleeve, Gauntlet and Glove, together or separately, help maintain the limb size of lymphedema patients. These comfortable garments provide gradient compression to assist in patient management following trauma, surgery or radiation therapy.




KEY FEATURES

- Fits a wide range of arm and hand shapes and available with or without a silicone band
- Two armsleeve lengths for optimal fit
- Fine seams with ultra thin and smooth compression fabric
- Glove is trimmable to fit finger length


COLORS



STYLES

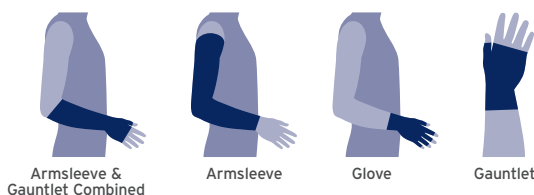

COMPRESSION CLASSES

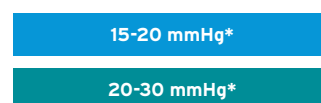

*All compressions shown are the mean compression for an average forearm size (armsleeve) or mean compression for an average palm size (glove / gauntlet). Not all colors and styles may be available in all compression ranges.

**Available in select styles.

JOBST® BELLA™ STRONG

Comfortable, Cost-effective Compression

JOBST® Bella™ Strong upper extremity garments are the ready-to-wear solution for patients who need to manage their lymphedema, but do not need the containment provided by a flat-knit custom garment. Bella™ Strong makes JOBST® quality available to a large number of patients by offering a wide range of sizes, lengths, different compression classes, and more accessible pricing.



KEY FEATURES

- Covers wide range of patient dimensions
- Easy to don, armsleeve comes with donning device
- Comfort Flat wrist seam reduces rolling
- Cotton-like feel
- The Bella™ Strong glove utilizes advanced flat-knit technology


COLORS



COMPRESSION CLASSES

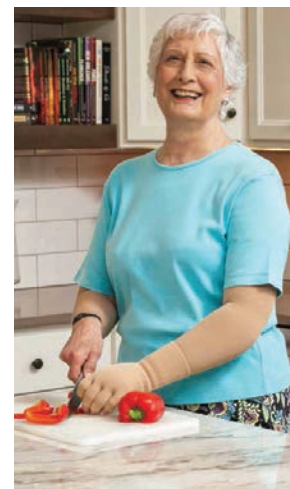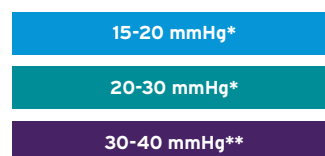

STYLES

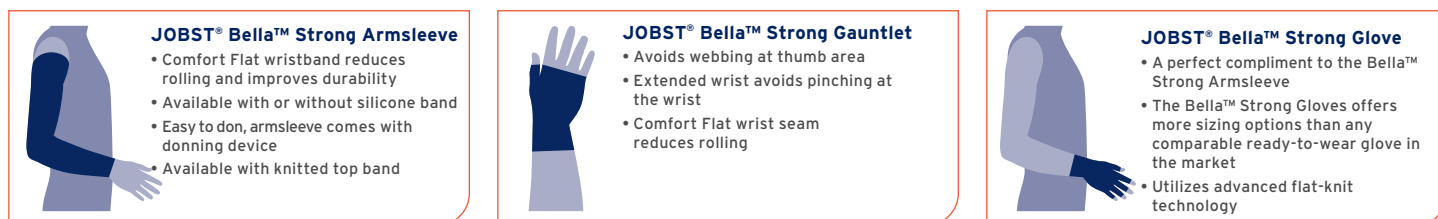

*All compressions shown are the mean compression for an average forearm size (armsleeve) or mean compression for an average palm size (glove / gauntlet).

**Bella™ Strong Gauntlet and Glove styles not available in 30-40 mmHg.

DAYTIME GARMENTS

JOBST® CUSTOM SEAMED

Nearly Limitless Solutions for Unique Challenges

JOBST® Custom Seamed custom-made garments were the first of their kind to manage lymphedema, and while much has changed in the past 71 years, our ability to find solutions to unique challenges has not. Custom Seamed garments are available for every part of the body and in countless configurations. Every garment is precisely cut and sewn by hand in Hickory, North Carolina.

KEY FEATURES

- Durable fabric for long lasting wear
- Precise gradient compression based on computer design
- Variety of options: zippers, linings, silicone bands and more

COLORS



COMPRESSION CLASS



STYLES

 Mitten to Wrist	 Mitten to Elbow	 Gauntlet	 Glove to Wrist	 Armsleeve (Glove to Elbow)	 Elbow Band	 Armsleeve (Wrist to Axilla)	 Armsleeve with Attached gauntlet	 Face Mask	 Chin Strap
 Sleeveless Vest	 Long-sleeved Vest	 Short-sleeved Vest	 Torso Band	 Body Brief with Sleeves	 Sleeveless Body Suit with Legs	 Panty Girdle (Above Knee, Closed Pubis)	 Panty Girdle (Below Knee, Closed Pubis)	 Abdominal Pelvic Garment	 Foot Glove (to Ankle)
 Ankle Garment	 Knee High	 Thigh Band	 Thigh High	 Waist High	 Stump Garment	 Waist High Stump Garment			

DAYTIME GARMENTS

JOBST® SOFTFIT™

Our innovative JOBST® SoftFit™ technology is a unique system of silicone yarns knitted into the welt of our compression garments. It provides gentle, seamless friction on the skin, decreasing the chance of constriction and slipping down.

LOOKS GOOD, FEELS GOOD

With the SoftFit™ band, garments are not only visually appealing and more comfortable to wear, they provide precisely the right amount of pressure, right where it is needed.

IMPROVED COMPLIANCE, IMPROVED QUALITY OF LIFE

Compression garments that feel comfortable lead to greater patient compliance.

MORE OPTIONS, HAPPIER PATIENTS

SoftFit™ Technology is offered in a wide variety of compression garments and styles.



BAND VARIATIONS

Product	Dotted Silicone	Lace Silicone	SoftFit	Sensitive	Ribbed Fleece Waistband, Adjustable Waistband	MicroDot	Floral Waistband
Elvarex® Upper Extremity	2.5 cm, on top, inside 5 cm, on top, inside		3 cm, on top (CCL 1 and 2 only)			5cm, on top	
Elvarex® Lower Extremity	2.5 cm, on top, inside 5 cm on top, inside 5 cm on top, waistband		5 cm, on top (Knee High only, CCL 1 thru 3)	5 cm, on top (Waist High only)	5cm, on top (Waist High only)	5cm, on top	
Elvarex® Soft Upper Extremity	2.5 cm, on top, inside 5cm, on top only		3 cm, on top			5cm, on top	
Elvarex® Soft Lower Extremity	2.5 cm, on top (Knee High only) 5 cm, on top		5 cm, on top (Knee High only)		(Waist High only)	5cm, on top	
Confidence	5 cm, on top (Thigh High only) 2.5 cm on top for Armsleeve		5 cm, on top (Knee High only) 2.5 cm on top for Armsleeve				(Waist High only)
Bella™ Lite Armsleeve	5 cm, on top						
Bella™ Strong Armsleeve	5 cm, on top						
Custom Bellavar	5 cm, on top (Knee & Thigh High) 2.5 cm, on top (Knee High only)	6 cm, on top (Thigh High only)					
Custom Seamless Soft	5 cm, on top (Knee & Thigh High) 2.5 cm, on top (Knee High only)	6 cm, on top (Thigh High only)	5 cm, on top, (Knee High only) (CCL 1 and 2 only)	5 cm, on top, (Thigh High only)			
Custom Seamed	2.5 cm, on top 5 cm, on top			5 cm, on top			

DAYTIME GARMENTS

FUNCTIONAL ZONES FOR JOBST® ELVAREX®, ELVAREX® SOFT & CONFIDENCE

Unique and innovative Functional Zones are created by the use of technologically advanced knitting techniques and specialized yarn selection. Functional Zones increase patient comfort through increased flexibility and breathability, without compromising medical effectiveness.

JOBST® FUNCTIONAL ZONES

- Use of highly specialized yarn and knitting techniques.
- More comfortable wearing due to increased breathability and flexibility.
- Special zones combat wrinkling when bending and speed moisture dissipation.



NIGHTTIME GARMENTS

JOBST® RELAX

Redefining Lymphedema Night Care

JOBST® Relax is the revolutionary nighttime garment that is designed for more than just sleeping. Its monofilament spacer fabric technology is unique in the marketplace and creates a supremely lightweight, flexible, and slim-line garment. Relax brings exceptional quality of life benefits and style to patients with mild to moderate lymphedema through the use of advanced knitting techniques and CoolMax® yarns. Because Relax is so breathable and flexible, patients can remain compliant through the evening hours.




KEY FEATURES

- Custom-made, flat-knit technology provides gradient compression therapy and stays in place
- Textured surface and monofilament spacer combine to create differential pressure areas that support stimulation of lymph flow while a patient is at rest


COLORS




STYLES

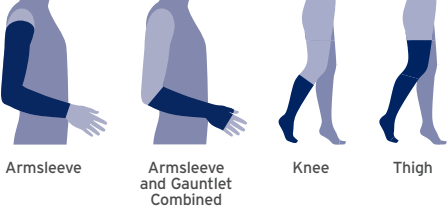

COMPRESSION CLASSES

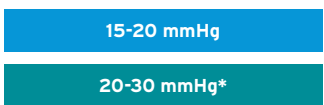

* 20-30 mmHg compression is only available in lower extremity.

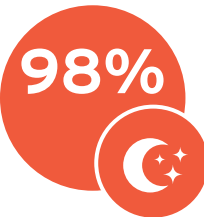


BETTER QUALITY OF SLEEP:

98% of patients rated quality of sleep **good or very good** compared to when wearing previous night-time products.



EDEMA CONTROL:

Many patients reported a major improvement in their lymphedema the next morning, with 90% of patients describing handling of day compression to be **good or very good** after using JOBST® Relax at night.

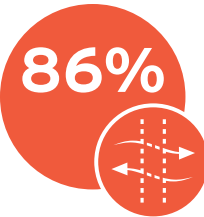

BREATHABILITY:

86.8% rated JOBST® Relax as **good or very good** in terms of avoiding heat generation and perspiration, respectively.



NIGHTTIME GARMENTS

JOBST® JOVIPAK®

Handmade Lymphedema Night Care

JOBST® JoViPak® garments are ideal for moderate to severe lymphedema patients who want an easy and comfortable alternative to nighttime bandaging. Unique continuous flow channeling and blended density chip foam construction allow JoViPak® garments to address even the most difficult edema and fibrotic tissue. Whether in ready-to-wear or custom configurations for every part of the body, each garment is handmade with care in Kent, Washington.

KEY FEATURES

- Variety of styles, sizes, and colors
- Custom and Ready-to-Wear
- Custom channeling options available
- Styles for virtually every part of the human anatomy
- Products to address any type of edema and fibrotic tissue
- Highly durable for increased longevity
- Uses up to four kinds of foam with no unpleasant chemical odors

COLORS

Cotton




Polartec® Power Dry®



PowerNet



Super PowerNet

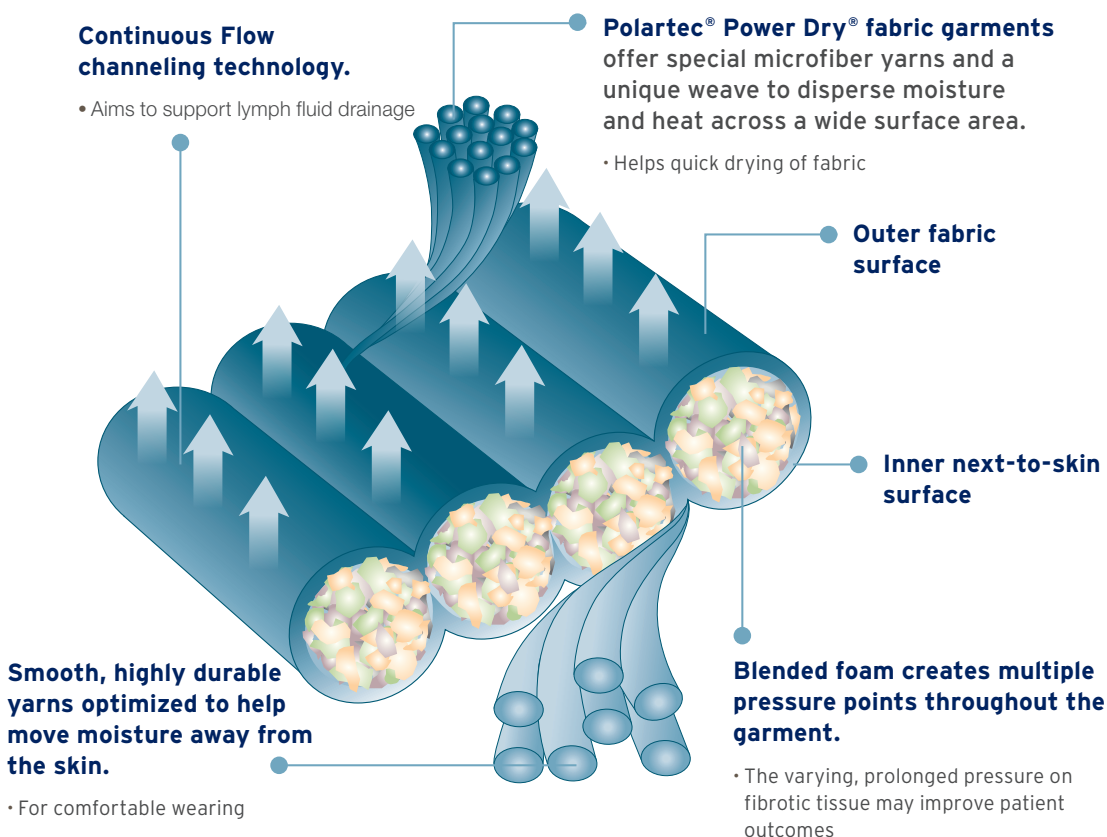

The **Polartec® Power Dry®*** fabric is also available in **Silk Weight**, a thinner version which is ideal for low profile slimline garments. **Silk Weight** is often recommended for patients having donning and doffing issues.

Garments made of **Organic Cotton/Lycra®*** may be a better choice for patients with a skin sensitivity to polyester and for products with **Velcro®** closures, as Velcro will snag the **Polartec® Power Dry®** fabric.

*Lycra® is a registered trademark of Invista, Polartec® and Power Dry® are registered trademarks of Polartec®, LLC.

NIGHTTIME GARMENTS

Head and Neck

- Standard & Extended Chin Strap
- Half & Full Face Mask
- Neck Pads
- Eye Pad

Torso

- Shoulder-Torso Sleeve
- Vest
- Busti

Genital area

- Boxer
- Boxer Capri
- Female & Male Genital Pads
- Custom Leg (DG1)

Upper Extremities

- Classic Arm Sleeve
- Two-Piece Arm Sleeve
- Stitched Finger Glove / Palm Pad

Lower extremities

- Classic Full Leg
- Classic Lower Leg
- Innaboot Full Leg & Lower Leg
- Hip-Hugger
- Custom Leg (AF1)
- ADVI

JOBST® JoViPads and JOBST® PitPaks can be used in treatment on any area of the body.



NIGHTTIME GARMENTS

BELLISSE®

Supporting Healing After Surgery

Bellisse® is uniquely designed to promote healing after breast surgery by providing uniform, gentle compression around the torso. When used in conjunction with JOBST® JoViPads®, Bellisse® encourages drainage of edema in the chest walls, which can aid in recovery.

KEY FEATURES

- Ready-to-Wear and available in a large range of sizes
- Unprecedented adjustability with both a front zipper for easy donning and a variable position back closure
- Attractive shape that can be worn comfortably day and night
- Soft to the skin, smooth while sleeping and unobtrusive under clothing
- Made of Polyester / Nylon / Spandex



Bellisse® is exclusively distributed by JOBST®

COLORS



SIZING RANGE


32 A/B - 50 DD/E*

*Please check order form with measuring instructions to determine the correct size.



BREAST AND CHEST WALL PADS

Breast and Chest Wall Pads are comfortable, foam-filled pads that conveniently fit into the Bellisse® to address specific areas of breast and chest wall fibrosis.

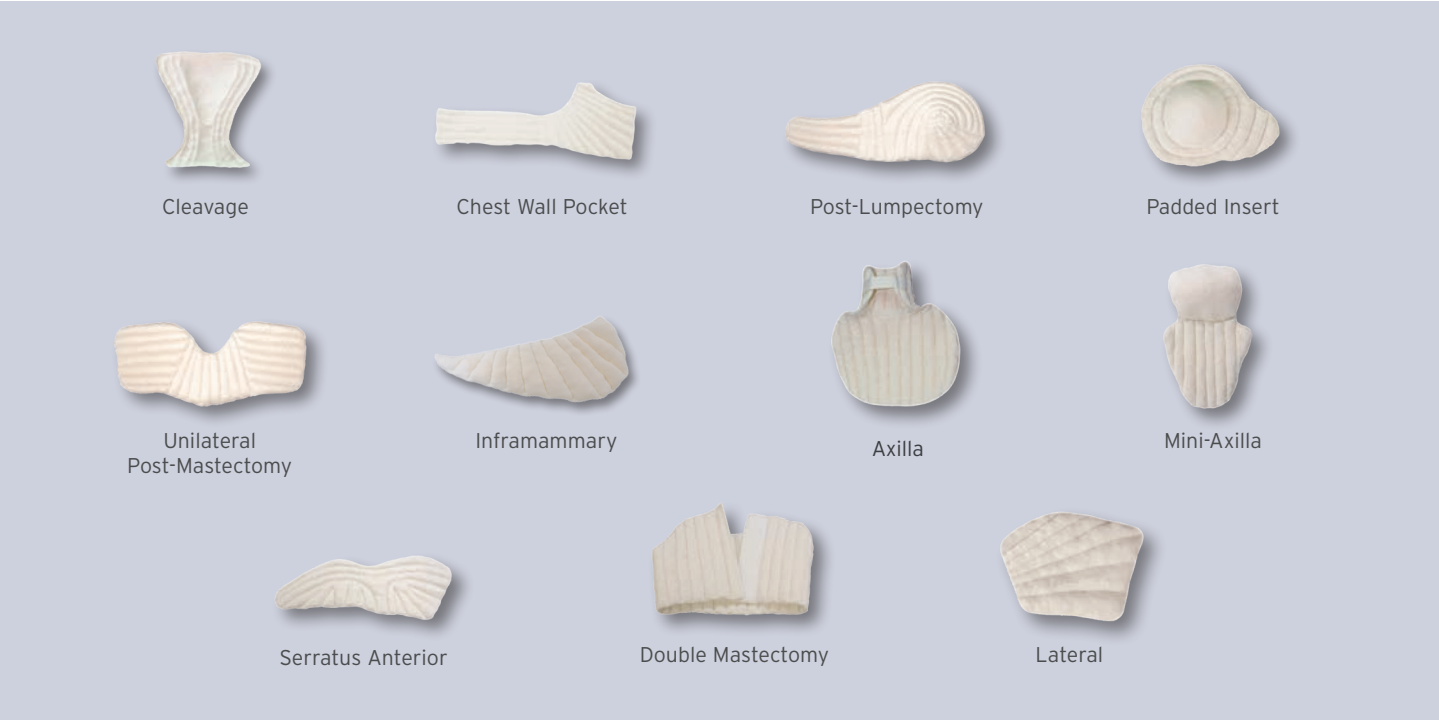

NIGHTTIME GARMENTS

JOBST® JOVIPADS®

JOBST® JoViPads® can be used in any area where there is a need for softening fibrotic tissue, providing pressure equalization, filling an anatomical indentation, or adding protection between skin folds.



JOVILINER ARM



Extends from the Distal Interphalangeal Joints (DIPs) to the axilla with stitching between each finger. Liners to be used under short stretch bandaging or pump appliances.


JOVILINER LEG



Full Leg has Continuous Flow Channels from the tip of the toes to the groin. Liners to be used under short stretch bandaging or pump appliances.

JOBST® PITPAKS

JOBST® PitPaks are filled with sterilized cherry pits and were specially designed to help address more difficult areas of fibrosis. These are an aggressive type of therapy and should only be undertaken with close therapist supervision.

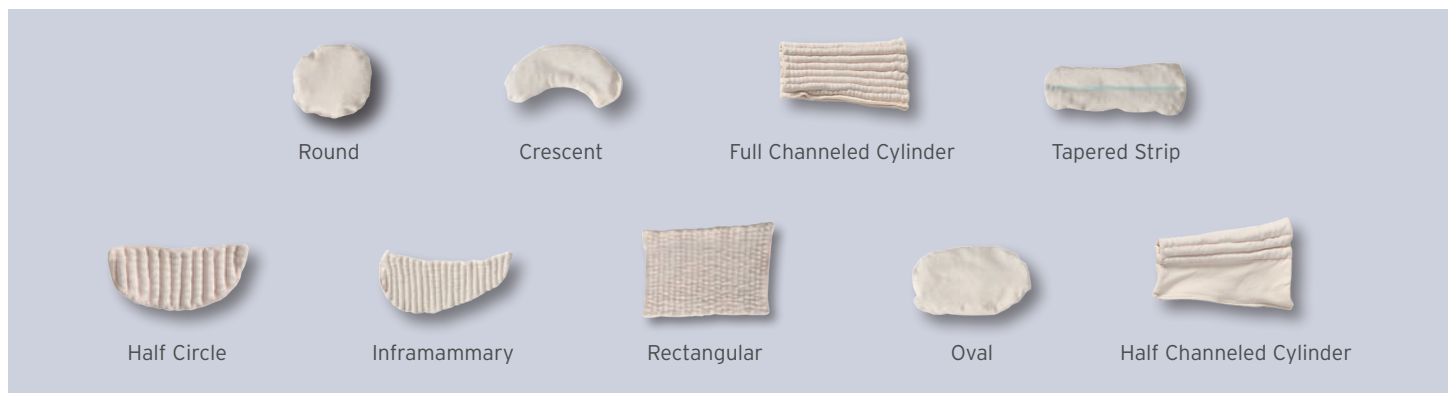

ACCESSORIES



JOBST® Donning Gloves*

Putting on JOBST® compression garments is always easier with JOBST® Donning Gloves! They help create friction, provide grip and help to evenly distribute the fabric on the leg.



JOBST® Stocking Donner

Easy to use device to help apply JOBST® compression garments. Ideal for people who have limited dexterity or have difficulty reaching their feet. Use with closed and open toe styles. Available in 2 sizes: Regular and Large Calf.



Dycem®

This 8" x 11" Dycem® is placed on the floor or table top to provide a non-slip surface the patient can push against as they don either an arm or leg garment. Dycem® can be easily cleaned with soap and water to renew its non-slip surface.



Arion Application Aids

Arion Application Aids assist in the donning and doffing of upper and lower extremity garments. These application aids can be used for both open ended and closed ended garments, and can be very helpful for patients with tenderness or weakness in their donning limb or are not strong enough to don the garment themselves.



JOBST® Roll-On Adhesive

- Designed to keep JOBST® Hosiery in place
- Ideal for thigh high lengths that do not have a silicone band or when a garter belt is not an option
- Helpful Hint: For flat-knit garments, apply in vertical strips



Cutimed® ACUTE

Cutimed® ACUTE 10% Urea Cream Mousse is an effective moisturizer for very dry skin. It can be applied either to specific areas or to the entire body. Reduces roughness of the skin and is suitable also for patients with very dry skin due to diabetes.

- Moisturizing
- Fast absorption
- With natural moisturizer

*Caution: This product contains natural rubber latex which may cause allergic reactions.

Customer Service Hours

Monday - Friday
8:00am - 6:00pm ET
(+1) 800 537 1063

Contact Information

Bellavar, Custom Seamless Soft, Elvarex®, Confidence & Relax Orders	HMS-Elvarex-Orders@essity.com
Bellavar, Custom Seamless Soft, Elvarex®, Confidence & Relax Returns	HMS-Elvarex>Returns@essity.com
FarrowWrap®	Farrow.Orders@essity.com
JoViPak® Custom	Info.JoViPak@essity.com*
JoViPak® Ready-to-Wear	Jovi.Orders@essity.com
Custom Seamed	SeamedOrders@essity.com*

*Hours may vary

Available On-Line 24 hours per day, 365 Days per Year

Standard Products: eshop.jobst-usa.com

Custom Products: eshop.jobst-usa.com

Sorbact® is a registered trademark of ABIGO Medical AB.
Bellisse® is a registered trademark of Lightning2, LLC.
VELCRO® is a registered trademark of Velcro BVBA.
CoolMax® is a registered trademark of INVISTA.
Dycem® is a registered trademark of Dycem Ltd.
Easy-Slide® is a registered trademark of Arion International.

The BSN Medical Inc., Reimbursement Disclaimer:

The Centers for Medicare & Medicaid Services (CMS) uses Palmetto GBA LLC., as the Pricing, Data Analysis, and Coding (PDAC) Contractor. The PDAC Contractor maintains the Product Classification List, located at <https://www4.palmettogba.com/pdac.dmecs/>. The Product Classification List is a searchable database containing products that have received HCPCS coding verification from the PDAC and their corresponding HCPCS Codes. BSN Medical Inc., however, makes no representations as to the accuracy of the information contained within the PDAC Database, nor any representations as to whether its products are reimbursable under any government sponsored healthcare program and/or private-insurance program. It is the provider's sole responsibility, in consultation with the insurer, if necessary, to determine medical necessity, ensure coverage criteria is met, and submit appropriate codes, modifiers, and charges for services rendered. RV: 12/24

Please refer to the product label and / or package insert for full instructions on the safe use of these products.



JOBST®,
an Essity brand



/JOBSTUSA



@JOBST_USA



@JOBSTforUSA



jobst-usa.com

BSN Medical Inc., an Essity company
5825 Carnegie Blvd., Charlotte, NC 28209-4633
Tel. (+1) 704 554 9933 Fax (+1) 800 835 4325
To order toll-free: Tel. (+1) 800 537 1063

52209 R19 ©2025 BSN Medical Inc. G25