# TENA SmartCare Change Indicator<sup>TM/MC</sup>

Professional caregivers  
Instructions for use

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Version: NA 2022-09-27

This guide is intended for professional caregivers, super users and managers using TENA SmartCare Change Indicator™/MC in a care home environment.

TENA SmartCare Change Indicator is a digital health technology solution that helps caregivers decide when to change absorbent products without the need for frequent manual checks. Besides being less intrusive, it also helps provide more effective, individualized care.

The Change Indicator attaches to the outside of a TENA absorbent product and tracks the urine saturation levels within. Caregivers and super users then receive information about the different saturation levels on their handheld devices via the TENA SmartCare Professional Care app. The TENA SmartCare Change Indicator system tracks urine saturation, but does not detect fecal episodes.

**TENA SmartCare Change Indicator is a system consisting of:**

- The TENA SmartCare dashboard, which serves as an information hub for caregivers, super users and managers.
- An app named TENA SmartCare Professional Care that is installed on the handheld devices of caregivers and super users.
- A reusable Change Indicator that consists of two parts: a sensor strip and a transmitter.
- A gateway that relays data from the Change Indicator to a secured server.

**In this guide, you will find instructions for how to use:**

The TENA SmartCare Change Indicator system, including TENA SmartCare transmitter, TENA SmartCare gateway, TENA SmartCare sensor strip, TENA SmartCare Professional Care app and TENA SmartCare dashboard.

**Symbols used in this guide**

**WARNING** Special care should be taken to ensure safety and efficiency. Failure to do so could cause damage to equipment or lead to the receipt of incorrect data.

**NOTE/ TIP** Important information or advice/ tips.

**CHECK** Important information for the system to function.

<b>Important information</b>	<b>4</b>
<b>Warnings</b>	<b>5</b>
<b>TENA SmartCare Change Indicator™/MC components</b>	<b>7</b>
<b>Getting started</b>	<b>8</b>
<b>Dashboard and setup</b>	<b>9</b>
<b>Download the app and set up the account</b>	<b>10</b>
<b>Set up the gateway</b>	<b>11</b>
<b>Register the transmitter code</b>	<b>12</b>
<b>Install the battery &amp; assemble the Change Indicator</b>	<b>13</b>
<b>Attach the Change Indicator</b>	<b>14</b>
<b>Use the Change Indicator</b>	<b>15</b>
<b>Remove the Change Indicator</b>	<b>16</b>
<b>Intended use</b>	<b>17</b>
<b>List of supported TENA products</b>	<b>18</b>
<b>Attachment instructions for supported TENA products</b>	<b>19</b>
TENA Briefs	19
TENA Belted briefs	20
TENA Underwear	21
<b>Replace the sensor strip &amp; the battery</b>	<b>22</b>
<b>Cleaning</b>	<b>23</b>
<b>Gateway lights &amp; buttons</b>	<b>24</b>
<b>Transmitter lights &amp; buttons</b>	<b>25</b>
<b>Troubleshooting</b>	<b>26–30</b>
<b>Technical description</b>	<b>31</b>
Operating characteristics	31–32
Electromagnetic compatibility (EMC)	33
Lifetime	33
Disposable parts	34
Labelling symbols	35

- Ensure you have received training and/ or have read and understood these instructions for use before using the product. Save these instructions for use for future reference.
- The Change Indicator should be attached to the outside of the incontinence product.
- The Change Indicator is not intended to be used in direct contact with the skin.
- The same Change Indicator should never be used on more than one resident.
- In the event of system failure or anticipated system failure, revert to a manual changing routine.
- TENA SmartCare Change Indicator<sup>TM/MC</sup> is intended for 24/7 use.
- The system provides information on urine saturation levels to support decision making regarding when TENA incontinence products should be changed.
- TENA SmartCare Change Indicator does not replace normal safety checks and routines for repositioning residents, checking for feces, administering medication, etc.
- Do not attach the Change Indicator to the inside of the incontinence product. TENA SmartCare Change Indicator should only be attached to the outside of the incontinence product.
- If you encounter any problems, equipment failure or if any other incidents occur, report them to the manufacturer or TENA SmartCare support.
- The Change Indicator is intended for use with TENA incontinence products.
- The adhesive strip for attaching the gateway will not adhere well to dirty, dusty or oily surfaces and may not adhere well to vinyl surfaces. Do not use tape on wallpaper, bare wood or on porous or painted surfaces.
- Only a CR2032 battery should be used in the Change Indicator. Using the wrong type of battery, including rechargeable batteries, can damage the Change Indicator.
- Do not use gateways with unapproved power supplies.

- Do not use the Change Indicator system on residents who experience frequent fecal incontinence. TENA SmartCare Change Indicator does not detect fecal episodes.
- Keep the Change Indicator and Change Indicator batteries away from children and pets.
- Seek immediate medical care if the battery is swallowed.
- If a defibrillator has been used on a resident who was wearing the Change Indicator, the Change Indicator must be replaced.
- Do not swing the Change Indicator in the direction of people, pets or nearby objects. Be aware of the risk of strangulation with the sensor strip.
- Clean the Change Indicator when fully assembled to prevent water from entering the device. Do not submerge in water or clean in a washing machine. Do not put the Change Indicator in an autoclave.
- Ensure adequate disinfection procedures are followed for residents with infectious diseases.
- If a resident is experiencing loose bowel movements or diarrhea, wait until the condition has been treated before resuming use of the Change Indicator.
- The Change Indicator and gateway should not be used in an oxygen-rich environment.
- The Change Indicator and gateway should not be used in the presence of flammable anesthetics.
- The Change Indicator is not intended for use in an aseptic environment.
- The Change Indicator is not magnetic resonance safe and should not be brought into a magnetic resonance imaging environment.
- Do not use the Change Indicator where X-Rays are performed.
- Do not attempt to repair or modify the Change Indicator or a gateway by yourself.
- If any part of the Change Indicator system appears to be damaged or broken, it should not be used. Contact TENA SmartCare support in your country.
- Do not expose the battery to high temperatures, disassemble it, damage it mechanically or put battery into the fire as this can cause an explosion or a fire.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

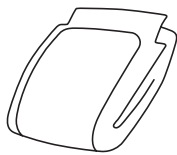
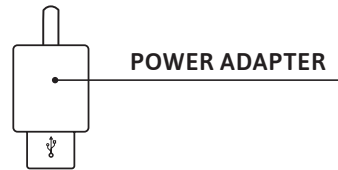
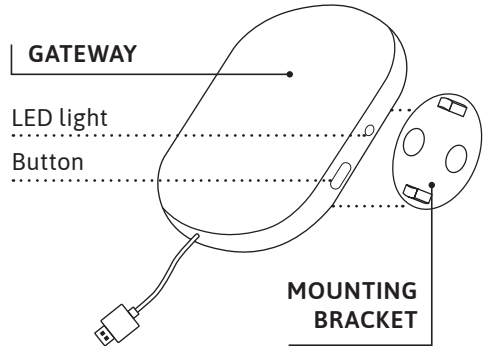
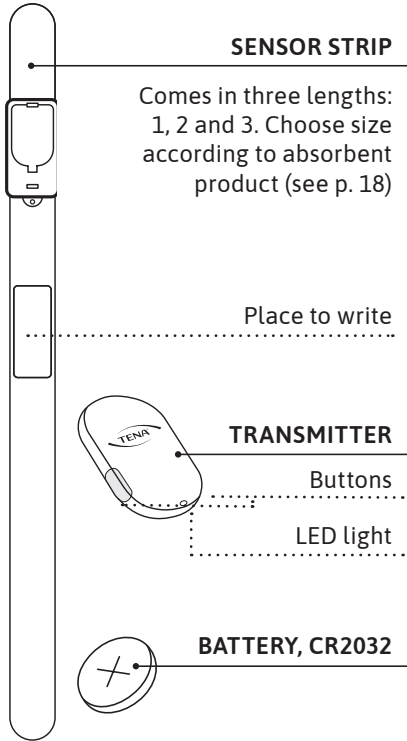
- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate this equipment.


This device contains license-exempt transmitter(s)/ receiver(s) that comply with Innovation, Science and Economic Development Canada's license-exempt RSS(s). Operation is subject to the following two conditions:

- This device may not cause interference.
- This device must accept any interference, including interference that may cause undesired operation of the device.

This equipment complies with the ICES RF radiation exposure limits set forth for an uncontrolled environment.

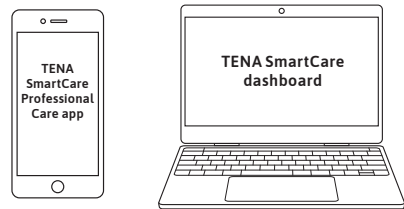A supported TENA absorbent product (see supported TENA products on p. 18 of this guide or in the TENA SmartCare dashboard).

A handheld device (iOS 11 and later or Android 7 or later) with TENA SmartCare Professional Care app and a computer to access the TENA SmartCare dashboard.


**CONSUMABLES:**

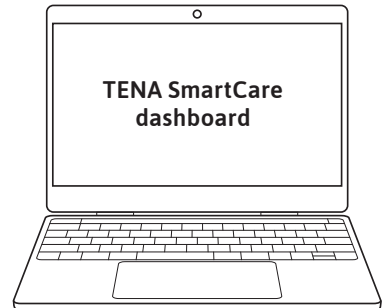Sensor strip 1 (short), sensor strip 2 (medium), sensor strip 3 (long), battery.

### TENA SmartCare dashboard

The dashboard serves as an information hub for super users and managers. It provides access to both real-time and past information, and enables super users and managers to:

- Check the system and staff performance.
- View statistics and create various reports.
- Edit system settings.

**The Home page** provides an overview of incontinence product changes throughout the care home. It shows the current status and statistics on residents, errors and response times for different units.

**The Residents page** provides head nurses and managers with an overview on the current status in a unit. Here, they can see more detailed information such as a resident status and resident history.

**The Statistics page** offers different ways to view data that is generated by the system to support decision making. It also provides insights regarding incontinence care received by residents and staff performance. Data can be filtered based on, e.g. units, time periods and shifts.

**The Setup page** allows head nurses and managers to update the system settings, such as number of units, resident information and specific system use settings.

**The Users page** allows nurses and managers to manage staff who are provided with access to the system.

## Functions in the dashboard

### OK

means the resident's absorbent product is dry.

### Some urine

means that the Change Indicator has identified urine in the product. The surface of the product is still dry and the product has not reached saturation.

### Consider change

means the resident's incontinence product is nearing capacity. Consider changing the incontinence product.

### Inactive

means the resident is inactivated for monitoring over a longer period of time.

**NOTE** The resident will not be monitored until you activate him/ her in the settings.

(Only visible in the dashboard)

### Off

means the resident's incontinence product is not being monitored by TENA SmartCare Change Indicator.

**NOTE** Resident will be back to monitoring automatically at the next shift.

### Error

means there is an error with either the resident's Change Indicator or the system. The caregiver should click on the icon to learn more.

See pp. 26–30 for more information on how errors can be resolved.

## Setup

Your TENA representative will show you how to access the TENA SmartCare dashboard and help set up users and residents in the system.

The app is a useful tool for caregivers. It provides real-time data and notifications on the residents they care for.

**Using the TENA SmartCare Professional Care app**

- 1 At the beginning of each shift, caregivers collect a handheld device and sign in to the TENA SmartCare Professional Care app to receive notifications. The password is provided by a super user or manager.
- 2 During their shift, caregivers receive notifications regarding urine saturation levels in residents’ absorbent products.

In the TENA SmartCare Professional Care app, beside each resident’s room number, caregivers will see a status icon for that particular resident. To protect residents’ privacy, they are identified by their room numbers rather than their names.

<p><b>102A</b> The resident’s bed number</p>	<p> <b>xx min</b> Time the resident has had this status for</p>	
<p> <b>OK</b> means the resident’s absorbent product is dry.</p>	<p> <b>Some urine</b> means that the Change Indicator has identified urine in the product. The surface of the product is still dry and the product has not reached saturation.</p>	<p> <b>Consider change</b> means the resident’s incontinence product is nearing capacity. Consider changing the incontinence product.</p>
<p> <b>Off</b> means the resident’s incontinence product is not being monitored by TENA SmartCare Change Indicator.</p>	<p> <b>Error</b> means there is an error with either the resident’s Change Indicator or the system. The caregiver should click on the icon to learn more. See pp. 26–30 for more information on how errors can be resolved.</p>	

**Setup**

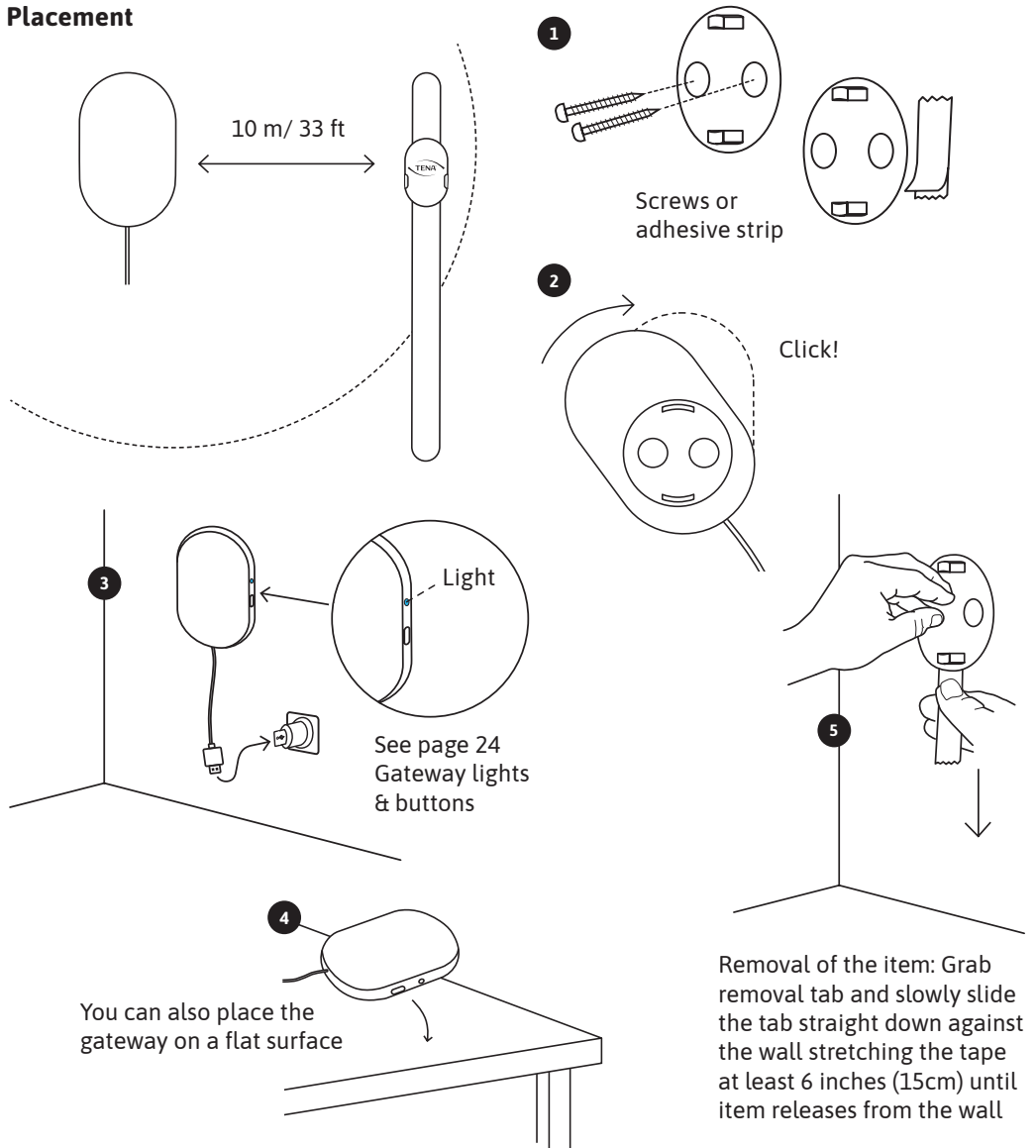
Your TENA representative will help you download and setup the TENA SmartCare Professional Care app.

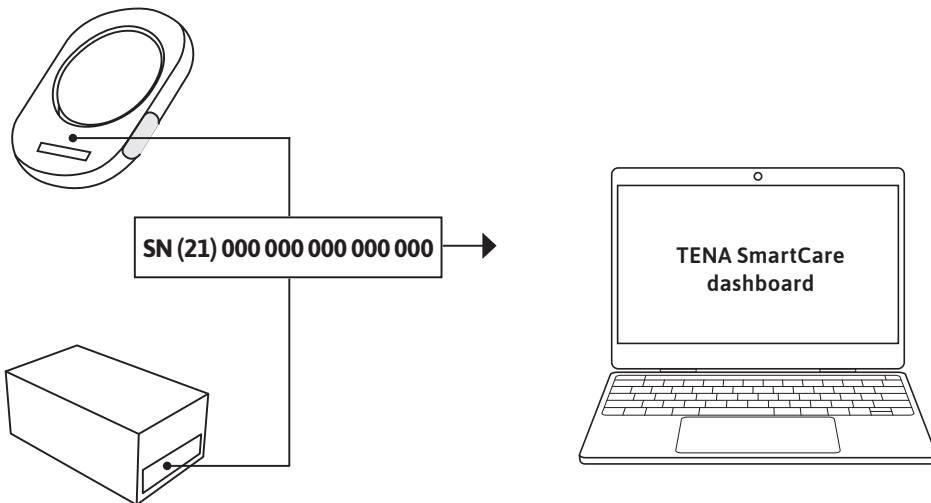
**NOTE**

The Change Indicator must be within 10 m/33 ft from a gateway to be monitored by the system.

**Placement**


**1** Find the transmitter code, which is located on the back of the transmitter and on the packaging.

**2** Enter the transmitter code in the dashboard.


**Install the battery**

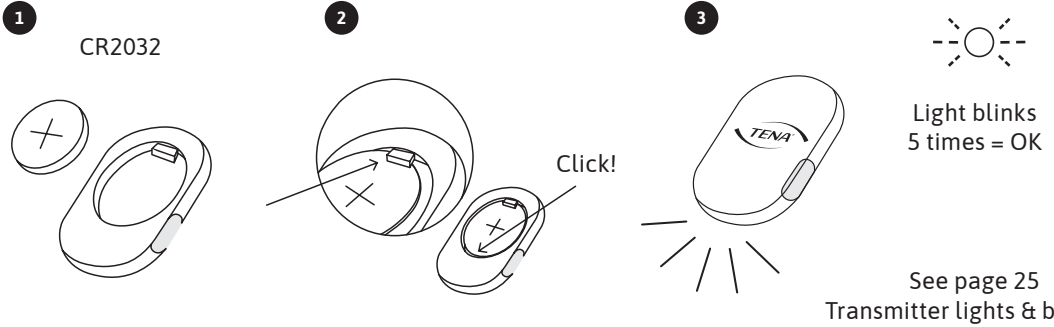**WARNING** Only CR2032 batteries shall be used in TENA SmartCare Change Indicator™/MC.


**Assemble the Change Indicator (transmitter + sensor strip)**

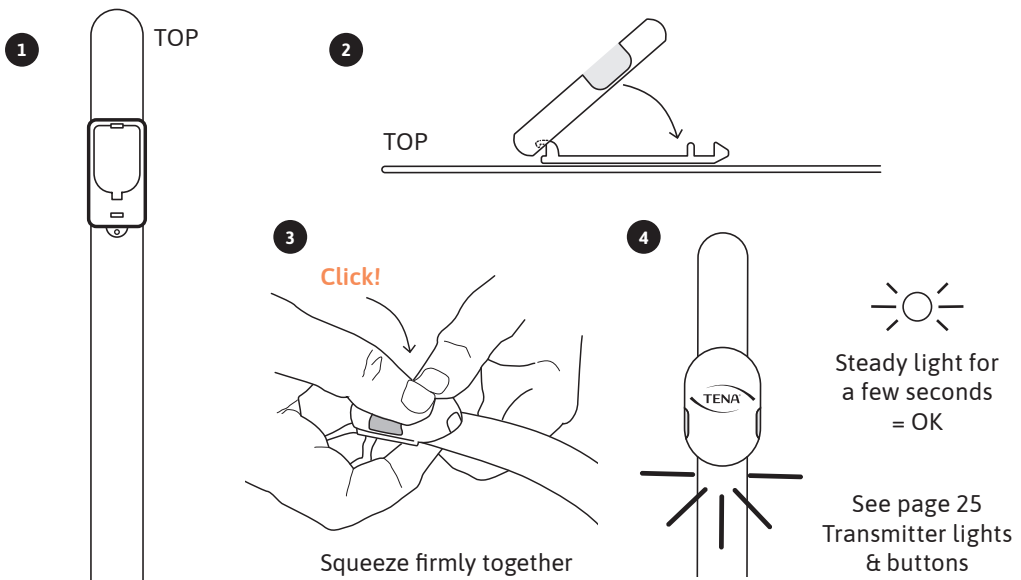See list of supported TENA absorbent products on p. 18 in this guide.


**1 Find the front**

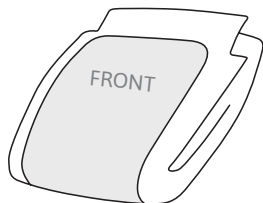
**2 Keep folded**

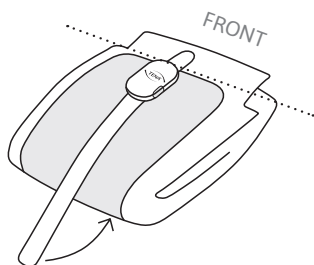**3 Align with the top of the thicker absorbent area**


**4 Place tightly**

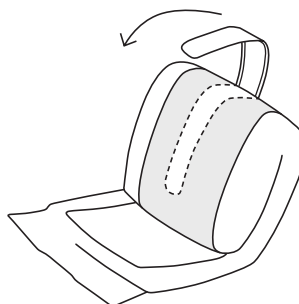
**NOTE**

Always attach the Change Indicator before you unfold the incontinence product. Start by placing the Change Indicator at the front of the incontinence product, on the thicker absorbent area, ensuring it is straight.

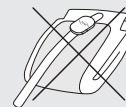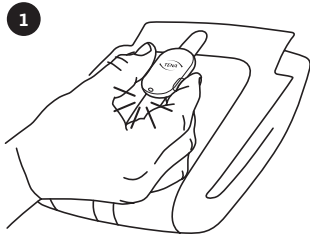Align the transmitter with the top of the thicker absorbent area of the incontinence product. Press firmly to ensure the Change Indicator has full contact with the whole incontinence product. If the user is very mobile, consider applying a fixation pant or ordinary underwear on top of the Change Indicator. This will keep it comfortably in place.


Press both buttons simultaneously for two seconds at every change to register the change in the system.



Steady light for a few seconds:  
The Change Indicator is ready to be applied to a resident

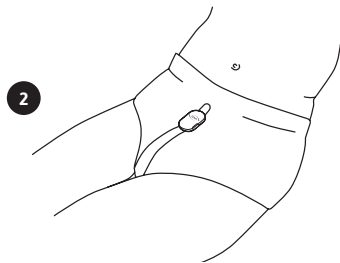Blinking light:  
Change battery, see p. 22

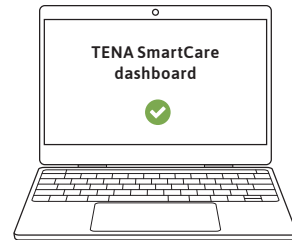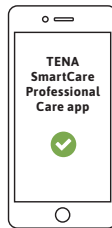Apply the TENA product with the Change Indicator on the resident

3

**OK**

No action needed.  
Resident's absorbent product is dry.

**Some urine**

No action needed. Some urine is detected, but the surface of the product is still dry.

**Consider change**

The resident's absorbent product is nearing saturation.

**Inactive (dashboard)**

The resident is inactivated for monitoring over a longer period of time.

**Off**

Resident's absorbent product is not being monitored by TENA SmartCare Change Indicator.

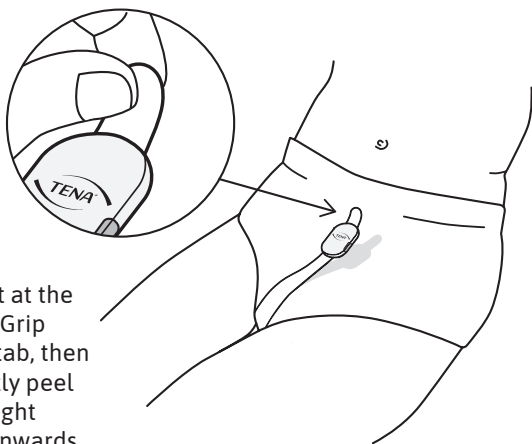
**Error**


See pp. 26–30 for more information.

## en REMOVE THE CHANGE INDICATOR

1

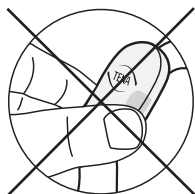Start at the top. Grip the tab, then gently peel straight downwards.


Don't grab and pull the transmitter.

### NOTE

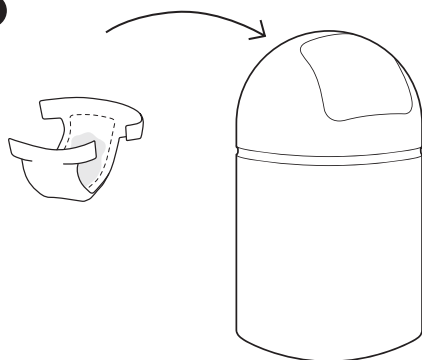The Change Indicator should be removed while the user is still wearing the incontinence product. Gently pull the Change Indicator down from the top, then roll the user onto his / her side before completing the removal.

2

Always make sure the Change Indicator has been removed before disposing of the TENA incontinence product.

**Intended use:**

TENA SmartCare Change Indicator<sup>TM/MC</sup> is an accessory to absorbent incontinence products, intended for use on individual(s) suffering from urinary incontinence in a home or professional environment who are dependent on one or more caregivers to change the absorbent incontinence products. TENA SmartCare Change Indicator estimates the degree of urine saturation in the absorbent incontinence product and notifies the caregiver(s). This facilitates the caregiver decision regarding when to change the absorbent incontinence product.

**Standard operation:**

- The Change Indicator should not be in contact with the user's skin during normal use. It should always be attached to the outside of the incontinence product.
- The operator of this Change Indicator is the caregiver of the incontinent user. The user wears the Change Indicator in combination with a TENA incontinence product.
- Attaching the Change Indicator to the incontinence product is considered a primary operating function.
- Only the caregiver is intended to interact with the Change Indicator.
- To turn monitoring off for the Change Indicator for a longer period, change the status to "Inactive" in the dashboard.
- The Change Indicator is intended for residents with healthy skin.

**Advanced operation:**

- Changing the battery and sensor strip are operations that should be carried out infrequently.

Using the TENA SmartCare Change Indicator according to its intended use does not add any risk compared to not using it.

The medical device complies with emission and immunity standards and regulations. It has been tested and fulfills requirements for both home and professional healthcare environments. The medical equipment is also compliant for professional healthcare home environment use.

## en LIST OF SUPPORTED TENA PRODUCTS

**CHECK** Ensure you have the correct sensor strip length for your TENA product. Only products with a textile-like outer lining can be used with the Change Indicator.

TENA® Absorbent Products	TENA® Productos absorbentes	Produits absorbants TENA®	S	M	R	L	XL	2XL	3XL
			S/M	M/R		L/XL			
			P	M		G	XG, TG	2XG, 2TG	
<b>Briefs</b>	<b>Pañales</b>	<b>Culottes</b>							
TENA Stretch™/MC: Ultra / Super				2		3		3	3
TENA®: Ultra / Super				2	2	3	3		
TENA Complete™/MC / TENA Complete + Care™/MC				2		2	2		
TENA Complete Ultra™/MC / TENA Complete + Care Ultra™/MC				2		2	2		
TENA® Small Brief	TENA® Pañal Pequeño	TENA® Culottes Petites	1						
TENA Bariatric Ultra™/MC							3		
<b>Underwear</b>	<b>Ropa interior</b>	<b>Sous-vêtement</b>							
TENA® Underwear: Plus / Extra / Overnight Super	TENA® Ropa Interior: Plus / Extra / Overnight™ Super	TENA® Sous-vêtement Absorbant: Plus / Extra / Overnight™ Super	1						
TENA ProSkin™/MC Protective Underwear for men / women	TENA ProSkin™ Ropa Interior Protectora para hombres / mujeres	TENA ProSkin™ Sous-vêtement pour hommes / femmes	1						
<b>TENA ProSkin™/MC Flex: Super, Maxi</b>			1	2		3	3		

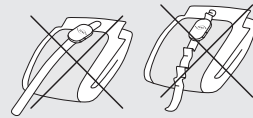
## TENA Briefs

TENA SmartCare Change Indicator™/MC should always be attached when the incontinence product is folded and before it has been applied to the resident.


### NOTE

Ensure the Change Indicator is straight. Smooth any sensor strip wrinkles, then press firmly to ensure the Change Indicator has full contact with the TENA absorbent product.

1

### Find the front

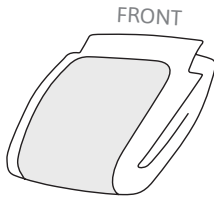
2

### Keep folded

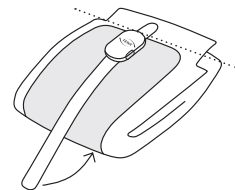
3

### Align with the top of the thicker absorbent area

4

### Place tightly

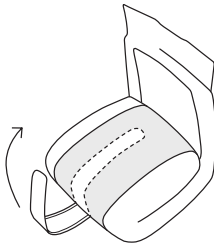
5

Every time you change an incontinence product, press both buttons simultaneously for two seconds to reset the transmitter and register the change in the system.

6

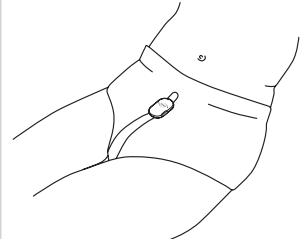Apply the absorbent product with the Change Indicator to the resident.






## TENA Belted briefs

TENA SmartCare Change Indicator™/MC should always be attached when the incontinence product is folded and before it has been applied to the resident.

### NOTE

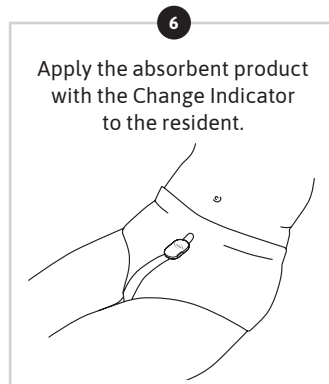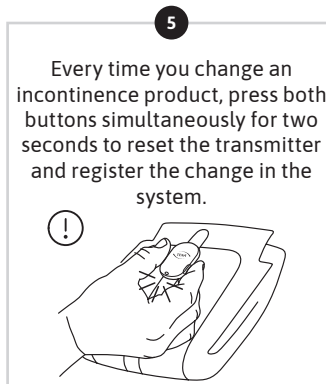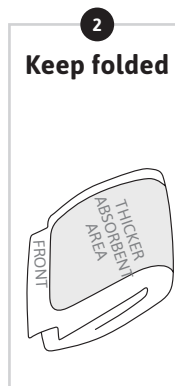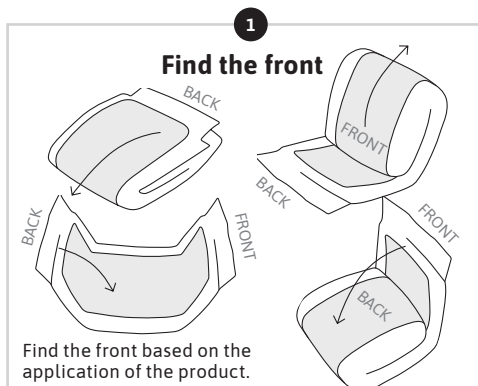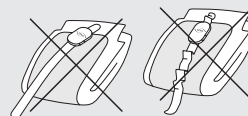Ensure the Change Indicator is straight. Smooth any sensor strip wrinkles, then press firmly to ensure the Change Indicator has full contact with the TENA absorbent product.


## TENA Underwear

TENA SmartCare Change Indicator™/MC should always be attached when the incontinence product is folded and before it has been applied to the resident.


### NOTE

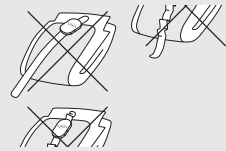Ensure the Change Indicator is straight. Smooth any sensor strip wrinkles, then press firmly to ensure the Change Indicator has full contact with the TENA absorbent product.

1

### Find the front

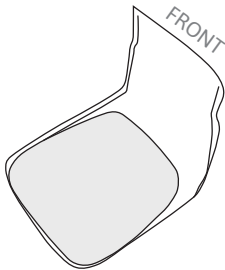
2

### Keep folded

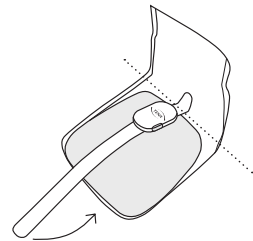
3

### Align with the top of the thicker absorbent area

4

### Place tightly

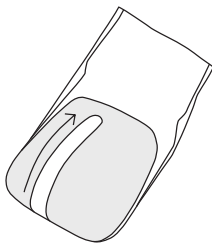
5

Every time you change an incontinence product, press both buttons simultaneously for two seconds to reset the transmitter and register the change in the system.


6

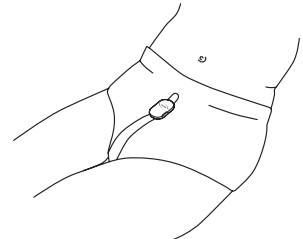Apply the absorbent product with the Change Indicator to the resident.


### Sensor strip replacement

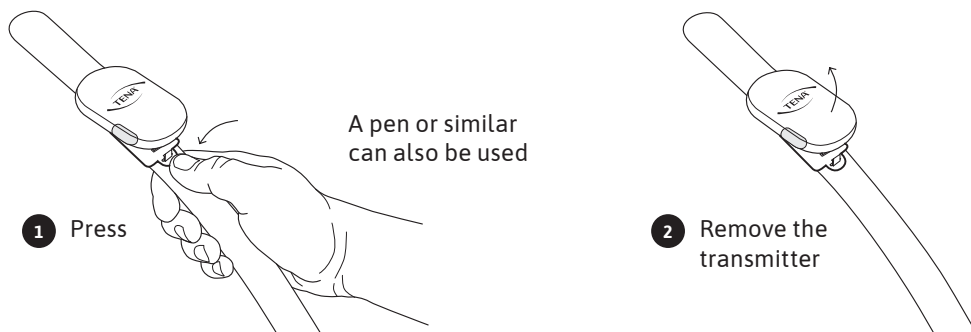The sensor strip is designed to withstand approximately 12 weeks of continuous use. Sometimes, however, the hooks on the back of the sensor strip can become covered with fibers or lint. This prevents the hooks from attaching to the incontinence product. If the fibers or lint cannot be removed, the sensor strip should be replaced. The sensor strip should also be replaced if it becomes soiled with feces and cannot be cleaned sufficiently. For cleaning advice, see p. 23 of this guide.

### Battery replacement

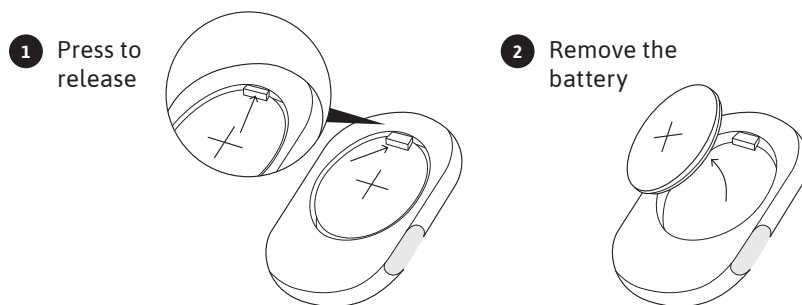If the LED light starts to blink, either the battery needs to be replaced or the transmitter is not properly attached to the sensor strip. The CR2032 battery will last for up to 12 weeks.


#### CHECK

If you have difficulty removing the battery, first press the locking tab, then tap the transmitter in your hand until the battery falls out. The transmitter is designed to withstand attachment and removal from a sensor strip four times per year over two years. With more frequent attachments/ removals, the system may not work as intended.

### Change Indicator

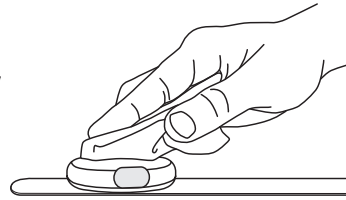The transmitter and sensor strip should never be cleaned separately – only clean the Change Indicator when it is fully assembled. Clean the Change Indicator using a cloth or a disinfectant when required.


We recommend using:

- Pre-soaked, alcohol-based wipes without additives.
- Alcohol-based disinfection foam for the hooks on the back of the sensor strip.


The assembled Change Indicator can be rinsed under a tap.

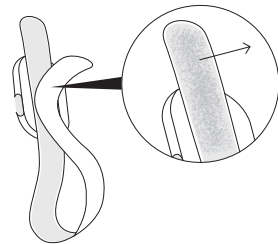Any fibers or lint attached to the hooks on the back of the sensor strip should be removed to ensure a secure fit between the Change Indicator and the incontinence product. Fibers can be removed using the free end of the sensor strip.

### Gateway

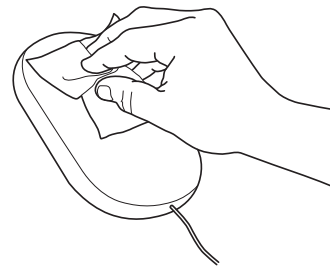Clean the gateway with a dry cloth when needed.


#### WARNING

Clean the Change Indicator when fully assembled to prevent water from entering the device.

Ensure adequate disinfection procedures are followed for residents with infectious diseases.


Do not clean in the washing machine.

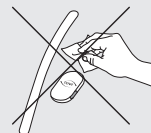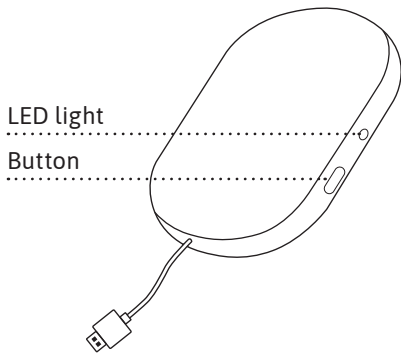Do not submerge in water.

### Gateway installation

After attaching the gateway to a power outlet:

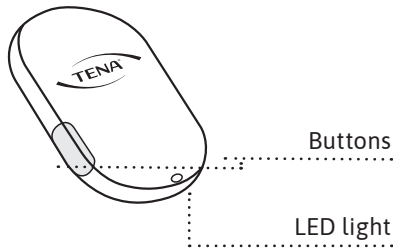
**NOTE**

The LED light can be difficult to see if exposed to a strong sunlight, for example. Try to block the sunlight to improve readability.

COLOUR LED LIGHT	WHAT IT MEANS
	Blinking blue light: Wait. The system is connecting. Allow up to 60 seconds for the light to start blinking.
	Steady green light: Everything is OK. Allow up to 60 seconds for the light to turn green.
	No light: Green light disappears after a while and the system goes into dark mode. Push the button to see if it has power and network connection. If no light appears, check the power connection.
	Steady red light: Error. See pp. 26–30 for more information.
	Blinking blue-green-red light: pull out the power. Wait 30 sec. Insert the power. Wait 2 min. If the error persists, contact TENA SmartCare support.

**NOTE**

The LED light can be difficult to see if exposed to a strong sunlight, for example. Try to block the sunlight to improve readability.

**Battery installation**

Once the battery has been installed in the transmitter:

LED LIGHT	WHAT IT MEANS
	Blinks five times: The battery is in the correct position. Attach the transmitter to the sensor strip.
	No light: OK The light disappears after a few seconds. Check for power by pressing both buttons simultaneously for two seconds.

**Change Indicator assembly**

Once the transmitter has been attached to the sensor strip:

LED LIGHT	WHAT IT MEANS
	Steady light for a few seconds: Everything is OK.
	Blinks five times: The transmitter is not attached properly or the battery is low. Try re-attaching the transmitter. If it does not help, change the battery.
	No light: OK The light disappears after a few seconds. Check for power by pressing both buttons simultaneously for two seconds.

**Applying the Change Indicator to the incontinence product**

Press both buttons simultaneously for two seconds at every change to register the change in the system.

LED LIGHT	WHAT IT MEANS
	Steady light for a few seconds: Everything is OK.
	Blinks five times: The transmitter is not attached properly or the battery is low. Try re-attaching the transmitter. If it does not help, change the battery.
	No light: Error. See pp. 27–31 for more information.

PROBLEM	SOURCE	SOLUTION
<b>Connection error (signal lost)</b>	Gateway	<p><b>Caregiver:</b></p> <ol style="list-style-type: none"> <li>1. Check power supply. Are the power cable and power adapter both properly attached?</li> <li>2. Press the gateway button. If there is a steady green light, everything is OK.</li> <li>3. If there is a blinking blue light, the gateway is trying to connect.</li> <li>4. If the gateway fails to connect and the light does not turn green after one minute, try moving the gateway to a position in the room where the network connection is stronger and wait for the green light.</li> <li>5. If this does not help, contact a super user at your care home and revert to a manual changing procedure for residents until the connection error is resolved.</li> </ol> <p><b>Super user:</b></p> <p>Repeat steps 1–4 above.</p> <ol style="list-style-type: none"> <li>5. Occasionally, the cellular network may go down. If this happens, inform the person in charge of the network at your care home.</li> <li>6. If this does not help, contact TENA SmartCare support.</li> </ol>
<b>Connection error (signal lost)</b>	Change Indicator	<p><b>Caregiver:</b></p> <ol style="list-style-type: none"> <li>1. Make sure the transmitter is properly attached to the sensor strip and that the battery is properly inserted (see p. 13 of this guide).</li> <li>2. Press both buttons on the assembled Change Indicator simultaneously for two seconds. A steady light should appear. If this does not happen, contact a super user at your care home and revert to a manual changing procedure for residents' incontinence products until the connection error is resolved.</li> </ol>

PROBLEM	SOURCE	SOLUTION
<b>Connection error (signal lost)</b>	Change Indicator	<p><b>Super user:</b></p> <ol style="list-style-type: none"> <li>1. Repeat steps 1–2 above.</li> <li>2. If there is no light, change the battery and reattach to a sensor strip. Press both buttons simultaneously again. If there is a steady light for a few seconds, everything is OK.</li> <li>3. If there is a blinking light, check that the transmitter is properly attached to the sensor strip. Adjust if needed. Press both buttons simultaneously again. If there is a steady light for a few seconds, everything is OK.</li> <li>4. If this does not help, replace the sensor strip. Press both buttons simultaneously again. If there is a steady light for a few seconds, everything is OK.</li> <li>5. If this does not help (no light or blinking light after attaching correctly to the sensor strip), consider changing the transmitter. When changing the transmitter, make sure to register the transmitter and assign it to the resident in the dashboard.</li> <li>6. If this does not help, contact TENA SmartCare support.</li> </ol>
<b>Connection error (signal lost)</b>	System range	<p><b>Caregiver:</b></p> <ol style="list-style-type: none"> <li>1. Both the resident and the Change Indicator should be in range of the gateway – usually within 33 feet/ 10 meters.</li> <li>2. If the resident is within range of the gateway, see “connection error” in the troubleshooting section of this guide.</li> <li>3. If this does not help, contact a super user at your care home and revert to a manual changing procedure for residents’ incontinence products until the connection error is resolved.</li> </ol>

PROBLEM	SOURCE	SOLUTION
<b>Connection error (signal lost)</b>	System range	<p><b>Super user:</b></p> <p>Repeat steps 1–2 above.</p> <p>3. If this does not help, contact TENA SmartCare support.</p>
<b>Change Indicator falls off</b>	Change Indicator	<p><b>Caregiver:</b></p> <ol style="list-style-type: none"> <li>1. Make sure you are using one of the supported TENA incontinence products with a textile-like outer lining. The Change Indicator does not attach to absorbent products with a plastic outer lining.</li> <li>2. Make sure you attach the Change Indicator to a folded product, straight from its plastic packaging. Do not attach the Change Indicator when the resident is already wearing the absorbent product.</li> <li>3. Check the hook-covered side of the sensor strip to see if it is covered with lint. Remove any lint.</li> <li>4. If this does not help, contact a super user at your care home and revert to a manual changing procedure for residents' incontinence products until the problem is resolved.</li> </ol> <p><b>Super user:</b></p> <ol style="list-style-type: none"> <li>1. Repeat steps 1–3 above.</li> <li>4. If this does not help, attach a new sensor strip. Make sure the new sensor strip is of the correct length (see p.18 of this guide).</li> <li>5 If this does not help, contact TENA SmartCare support.</li> </ol>

PROBLEM	SOURCE	SOLUTION
<b>The system notifies me too early / too late</b>	Status indication	<p>a) You receive the “consider change” notification when an absorbent product is still dry or has not been worn for long.</p> <p>b) You receive the “consider change” notification when an absorbent product is already too saturated with urine to be comfortable or the product has already been worn for too long.</p> <p><b>Caregiver:</b></p> <ol style="list-style-type: none"> <li>1. Make sure you are using one of the supported TENA products (see p. 18 of this guide).</li> <li>2. Make sure you are using the right sensor strip length for your TENA product (see p. 18 of this guide).</li> <li>3. Make sure the Change Indicator is attached correctly (see pp. 19–21 of this guide).</li> <li>4. If this does not help, contact a super user at your care home.</li> </ol> <p><b>Super user:</b></p> <ol style="list-style-type: none"> <li>1. Repeat steps 1–3 above.</li> <li>2. If this does not help, make sure the right TENA product with the right absorption level is assigned to this resident in the dashboard.</li> <li>3. If this does not help, please contact TENA SmartCare support.</li> </ol>

PROBLEM	SOURCE	SOLUTION
<p><b>I don't receive any notifications</b></p>	<p>Status indication</p>	<p><b>Caregiver:</b></p> <ol style="list-style-type: none"> <li>1. Check "connection error" in the troubleshooting section of this guide.</li> <li>2. If this does not help, contact a super user at your care home.</li> </ol> <p><b>Super user:</b></p> <ol style="list-style-type: none"> <li>1. Check "connection error" in the troubleshooting section of this guide.</li> <li>2. Check the dashboard. Is the proper transmitter assigned to the resident?</li> <li>3. If this does not help, please contact TENA SmartCare support.</li> </ol>

## Operating characteristics

The Change Indicator is classified as an internally powered device for continuous operation.

### Processing units

Transmitter & gateway

CHARACTERISTICS	VALUE/ RANGE
Wireless technology	RF communication, BLE
RF frequency	2.4 GHz
Max. output level (ERP)	0 dBm
Electrical emissions	Class B, level 47 dB mV/m
Operating voltage	2.7 – 3.3 V
Power consumption	Avg. 135 uW Peak 75 mW
Remote communication range	At least 10 meters/ 33 feet in open environment, without obstructions

### Battery module

TYPE	CAPACITY/ VOLTAGE
CR2032	220 mAh/ 3.0 V nominal

This value is with the recommended battery: CR2032

### Product component dimensions (mm/ inch)

ITEM	LENGTH	WIDTH	HEIGHT	DIAMETER
Transmitter	51.8 mm/ 2.04"	34.5 mm/ 1.36"	13 mm/ 0.51"	N/A
Sensor strip (1, 2, 3)	510 – 690 mm/ 20.08 – 27.16"	27.5 mm/ 1.08"	7.6 mm/ 0.30"	N/A
Battery	N/A	N/A	3.2 mm/ 0.13"	20 mm/ 0.79"

### Product weight (g/ ounce)

COMPONENT	WEIGHT
Transmitter	12.6 g/ 0.44 oz
Sensor strip (1, 2, 3)	12.6 – 15.9 g/ 0.44 – 0.56 oz
Battery	3.0 g/ 0.11 oz

### Environmental conditions

CONDITION	MINIMUM	MAXIMUM
Storage & transport temp.	+10 °C/ 50 °F	+40 °C/ 104 °F
Storage & transport humidity	65% Rh	90% Rh
Operating temp.	+15 °C/ 59 °F	+40 °C/ 104 °F
Operating humidity (non-condensing)	15% Rh	90% Rh
Pressure	700 hPa	1060 hPa

## Electromagnetic compatibility (EMC)

RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm/ 12" to the Change Indicator. Otherwise, degradation of the performance of this equipment could occur.

Interference may occur in the vicinity of equipment marked with this symbol.

## Lifetime

- Read best before date on accompanying packaging.
- Expected lifetime of the transmitter is two years and the sensor strip is 12 weeks, after the first use.
- Expected shelf life (from the manufacturing date) for the transmitter is three years and for the sensor strip is three years.
- The transmitter is designed to withstand attachment and removal from a sensor strip four times per year, for two years. With more frequent attachments/ removals, the system may not work as intended.

## Disposable parts

- Clean the Change Indicator before disposal.
- Follow local regulations for disposal of the Change Indicator, the gateway and the batteries.
- This declaration relates exclusively to the radio equipment in the state in which it was sold and excludes components that are added and/ or operations carried out subsequently by the final user.

### WEEE mark:

- This product has been designed with recycling, reuse and waste management in consideration. Essity will provide the necessary information to recycling facilities and government agencies as required by WEEE Directive. For more information, please contact the manufacturer at [www.tena.com](http://www.tena.com).
- The product will be labeled with the WEEE mark as shown on the side, in accordance with European Standard EN 50419.
- The proper disposal of WEEE products will conserve natural resources (through reuse and recycling) and protect environmental and human health through proper disposal.
- WEEE-labeled products must be kept separate and returned to a designated collection point for dismantling, recycling and/ or reuse.
- The separate collection and recycling of your waste equipment at the time of disposal will help to conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment. For more information about where you can drop off your waste equipment for recycling, please contact your local government, your commercial, institutional and/ or municipal waste disposal service or the location where you purchased the product.

## Labeling symbols

The following symbols may appear on your TENA SmartCare Change Indicator™/MC, accessories and/ or packaging:

Manufacturing date

**UDI:**

Unique Device Information

Manufacturer

Not MR safe

Instructions for use

European Conformity mark

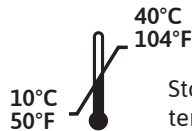
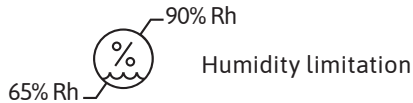
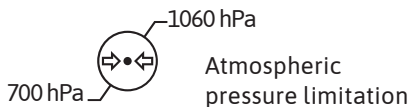
WEEE mark  
Dispose of electrical components in accordance with your local regulations

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Essity HMS North America Inc.